

ULTRA UNIVERSAL RESTORATIVE MATERIAL INSTRUCTIONS FOR USE

INTRODUCTION

aura bulk fill is a light cured, radiopaque, high strength, universal composite designed to meet most of your restorative needs. The **aura bulk fill** has a 5mm depth of cure which reduces the standard number of incremental layering when restoring deep cavities.

INDICATIONS

- Posterior restorations
- Core buildup
- Splinting
- Sandwich technique with glass-ionomer

CONTRA-INDICATIONS

- Pulp capping.
- Do not use in conjunction with any eugenol containing materials.
- Anyone with allergy to acrylics

PLACEMENT PROCEDURE:

1. Isolate tooth.
2. Prepare cavity using standard technique for composite resin restorations.
Note: In cases where pulp protection is necessary, use a calcium hydroxide liner.
3. Apply a dentin/enamel bonding agent such as SDI Go! as per manufacturer's instructions.
4. Place **aura bulk fill** no more than 5mm thick.
5. Light cure for 20 seconds using a high powered LED curing light (460 - 480nm wavelength).
6. Finish restoration using standard technique.

STORAGE AND HANDLING

- Store at temperatures between 10° and 25°C (50°-77°F).
- Use at room temperature between 20° and 25°C (68°- 77°F).
- Syringe system: when not in use replace the cap tightly.

PRECAUTIONS

- Avoid prolonged contact of **aura bulk fill** with the skin or oral tissue, as it may cause inflammation of the oral tissues or skin sensitization.
- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Do not expose material to direct light.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID

- **Eye (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol. Wash thoroughly with water.
- **Ingestion:** Rinse mouth thoroughly. Drink plenty of water/milk. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.