



# polapaint

E  
N  
G  
L  
I  
S  
H

## Paint-on tooth whitening system (8% carbamide peroxide) INSTRUCTIONS FOR USE

Pola Paint is an easy to use paint-on tooth whitening gel. It is a pleasantly flavored gel with a low viscosity which is easily applied onto the tooth surface. Before using Pola Paint, your dentist should assess your suitability for the product including your medical history and explain the application procedure.

### INDICATIONS FOR USE:

- Whitening of discoloured vital teeth.
- Whitening of discoloured non-vital teeth.

### COMPOSITION:

carbamide peroxide, ethanol, glycerol, additives, water, flavor

**NET WEIGHT:** 5.1g

### HYDROGEN PEROXIDE EQUIVALENT:

Pola Paint 8% Carbamide Peroxide = 2.7% Hydrogen Peroxide

### INSTRUCTIONS:

1. Brush and floss teeth as normal.
2. Blot dry the front surface of teeth.
3. Whilst retracting the lips, apply a thin layer of Pola Paint gel to the front surfaces of the teeth, re-loading brush as required.
4. After application do not let your lips touch your teeth for 30 seconds.
5. Leave Pola Paint gel on teeth for 30 minutes. Do not eat or drink during this time to maximize results.
6. At the end of 30 minutes, remove the Pola Paint gel by brushing with a tooth brush.
7. Use Pola Paint twice a day for 14 days.

For best results, one of the daily treatments should be applied before going to bed, allowing the Pola Paint to stay on overnight. When not in use, **firmly** replace cap to prevent gel from drying out.

### PRECAUTIONS:

- To be used under the supervision of a dentist.
- Do not use after expiry date.
- Use at room temperature.
- Pola paint should not be used by patients with known allergies or chemical sensitivity to peroxides, glycerol etc.
- Do not smoke during or immediately after the whitening process.
- Keep product away from heat and naked flame, as product is flammable.
- Keep out of reach of children and pets.
- Not to be used by pregnant or lactating women or children under the age of 14 years.
- Discard Pola Paint after treatment has been completed.
- MSDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.

### STORAGE:

- Store in a cool and out of direct sunlight at temperatures between 4°C - 20°C / 39°F -68°F
- DO NOT STORE IN CLOTHING POCKETS.

### FIRST AID:

- Skin (contact): Pola paint may be irritating to skin. Wash with running water if contact occurs. If any soft oral tissue irritation occurs, remove gel, rinse mouth with salty water or sodium bicarbonate solution and drink milk. Discontinue treatment and consult your dentist if there is persisting sensitivity or any reaction to the gel.
- Eyes (contact): Open eye and flush thoroughly with water for 15 minutes. Seek medical attention.



Made in Australia by SDI Limited  
Bayswater, Victoria 3153  
Australia 1 800 337 003  
Austria 00800 022 55 734  
Brazil 0800 770 1735  
France 00800 022 55 734  
Germany 0800 100 5759  
Ireland 01 886 9577  
Italy 800 780625  
New Zealand 0800 734 034  
Spain 00800 022 55 734  
United Kingdom 00800 022 55 734  
USA & Canada 1 800 228 5166  
[www.sdi.com.au](http://www.sdi.com.au)  
[www.polawhite.com.au](http://www.polawhite.com.au)

