

6 and 9.5% hydrog 10%, 16%, 18% and 22% carbamide p advanced tooth whitening system

INSTRUCTIONS FOR USE

d Pola Night are high viscosity, neutral pH tooth lels. Both are pleasantly flavoured, crystal clear sustain with a unique blend of soothers, conditioners and a ent high enough to further reduce sensitivity.

N: 3.0-9.5% wt Hyd < 47% wt Additi 30% wt Glycerol 20% wt Water 0.1% wt Flavour

Pola Night

10.0-22.0% wt Carb < 40% wt Additives 30% wt Glycerol 20% wt Water 0.1% wt Flavour

T WEIGHT: pending on kit purchased, 1.3g syringe ase refer to outside of box. 3g syringes

HYDROGEN PEROXIDE EQUIVALENT: ola Night 10% Carbamide Peroxide = 3 ola Night 16% Carbamide Peroxide = 5 ola Night 18% Carbamide Peroxide = 6 ola Night 22% Carbamide Peroxide = 7

TREATMENT GUIDELINES:			
	3%	6% & 7.5%	9.5%
Pola Day	2 or 3 x 30 minutes / day or 1 x 60 minutes /day	1 or 2 x 30 minutes / day or 1 x 45 minutes / day	2 x 15 minutes/day or 1 x 30 minutes / day
Pola Night	10%	16% & 18%	22%
	1 x 2 hours / day to overnight	1 x 90 minutes / day to overnight	1 x 45 minutes / day

ensitivity will require a day or two break Shortening treatment time is also suggeste

- ROCEDURE:

 Ill drop of gel into every compartment of the teeth undergoing treatment.

 y, with the gel around the teeth.

 excess gel in mouth with a tissue or dry soft brush.
 nent, remove tray. Rinse tray and mouth with











IOTE:

o not eat, drink or smoke during treatment. Do not smoke nmediately after treatment - wait for at least two hours. boods and drinks containing strong colours should be avoide or at least 48 hours or consumed in moderation. Use gel at boom temperature.

F). Do not use after

PRECAUTIONS:

rritant. Avoid contact with eyes or gums. Avoid swallowing.

To be used under the supervision of a dental professional.

keep out of reach of children and pets. Not to be used by oregnant or lactating women or children under 14 years old.
Tatients with a history of chemical allergies are advised to carry
sut allergy testing by a specialist before using these products.
Discontinue use if any unusual sensitivity or reactions occur and
sonsult your dentist.
To not inject Pola Day or Pola Night into the body.
So available at www.sdi.com.au or contact your regional
epresentative.

FIRST AID:
For advice, contact a Poisons Information Centre
(Australia 131 126, New Zealand 0800 764 766) or a doctor.
If swallowed, do not induce vomiting. If in eyes, hold eyelids
apart and flush the eye continuously with running water.
Continue flushing until advised to stop by the Poisons
Information Centre or a doctor, or for at least 15 minutes. If
skin or hair contact occurs, remove contaminated clothing and
flush skin and hair with running water. Seek medical advice if
symptoms persist.



Made in Australia by SDI L ayswater, Victoria 3153 australia 1 800 337 003 austria 00800 0225 5734 razil 0800 770 1735 rance 00800 0225 5734 iermany 0800 1005759 eland 00800 0225 5734

taly 00800 0225 5734 New Zealand 0800 734 034 Spain 00800 0225 5734 United Kingdom 00800 0225 5734 USA & Canada 1 800 228 5166 www.sdi.com.au

