SDI polazing
35% carbamide peroxide
advanced tooth whitening system
INSTRUCTIONS FOR USE
FOR PATIENTS AND
DENTAL PROFESSIONALS

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ESPAÑOL

Pola Zing is a 35% carbamide peroxide, high viscosity, neutral pH take home tooth whitening gel. It is a pleasantly flavoured, crystal clear sustained release gel with a water content high enough to reduce sensitivity.

INDICATIONS FOR USE:

- Whitening of discoloured vital teeth.
- Whitening of discoloured non-vital teeth.

COMPOSITION: 35% Carbamide peroxide
40% Water
0.5% Flavour
24.5% Thickening agents

NET WEIGHT:

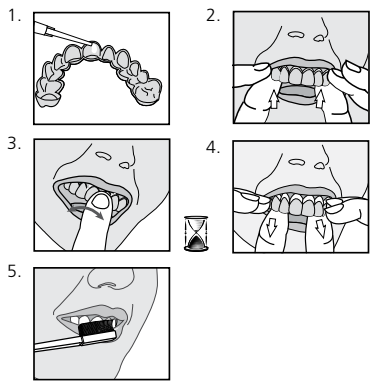
Depending on kit purchased, 1.3g syringes or 3g syringes. Please refer to outside of box.

HYDROGEN PEROXIDE EQUIVALENT:

Pola Zing 35% Carbamide Peroxide = 11.7% Hydrogen Peroxide

INSTRUCTIONS:

- Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
- Seat the tray, with the gel around the teeth.
- Wipe away excess gel in mouth with a tissue or dry soft brush.
- After 15 minutes of treatment, remove tray. Rinse tray and mouth with lukewarm water.
Note: Treatment time can be up to 30 minutes if required.
- Brush teeth.

STORAGE AND HANDLING:

- Store product in the refrigerator (2°-8°C / 35°-45°F).
- Do not use after expiration date.
- Use product at room temperature.

PRECAUTIONS:

- To be used under the supervision of a dentist.
- Do not eat, drink or smoke during treatment.
- Do not smoke immediately after treatment - wait for at least two hours.
- Food and drinks containing strong colours should be avoided or consumed in moderation during the entire treatment period.
- Keep out of reach of children and pets.
- Not to be used by pregnant or lactating women or children under 14 years old.
- Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using this product.
- Discontinue use if any unusual sensitivity or reactions occur and consult your dentist.
- Do not inject Pola Zing into the body.
- CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST.**
- MSDS available at www.sdi.com.au or contact your regional representative.

FIRST AID:

- Eye (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
- Skin/Tissue (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
- Ingestion (large amount):** Drink plenty of milk or water and seek medical advice.

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