INDICATIONS FOR USE:

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

NOTE: To insert the supplied powder scoop leveller into the powder jar, simply remove the moisture seal, then place gently by pushing into the jar. Do not remove the secondary moisture seal from the inside of the jar cap.

- INSTRUCTIONS FOR VITAL TOOTH WHITENING**
- Determine and record pre-operative shade. Please follow this Vita® shade guide arrangement according to degree of brightness: B1+A1+B2+D2+A2+C1+C2+D4+A3+D3+B3+A3.5+B4+C3+A4+C4
 - Clean teeth with **ONLY** a flour based pumice.
 - Place cheek retractors and then cover exposed lip surface with petroleum gel.
 - Dry teeth and apply Gingival Barrier to both arches, slightly overlapping enamel and interproximal spaces.
 - Light cure in a fanning motion for 10-20 seconds until Gingival Barrier is cured.
 - Mixing of whitening gel: Using the supplied empty clear pot (with black lid) as a mixing well, dispense the required amount of powder and liquid. Recommendations are listed in the below table. Please note that these are only recommendations, and that more or less liquid can be added to obtain a desired gel consistency. Replace jar/bottle caps tightly.

1 tooth, 2 applications
1 level scoop of powder
5±1 drops of liquid

Single arch, 1 application
2 level scoops of powder
10±2 drops of liquid

Dual arch, 1 application
4 level scoops of powder
20±3 drops of liquid

- Immediately mix using a brush applicator until gel is homogeneous.
- Apply a **thick** layer of gel to all teeth undergoing treatment. Leave gel on for 8 minutes (Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations)
- Suction off using a surgical aspirator tip.
- Complete Steps 6-10 three times (optional 4 times). Note: gel must be used within 20 minutes of mixing.
- After the last application, suction all the gel off, then wash and apply suction.
- Remove Gingival Barrier by lifting it from one end.

INSTRUCTIONS FOR NON-VITAL TOOTH WHITENING.

- Verify adequate endodontic obturation.
 - Isolate tooth with rubber dam.
 - Open lingual access to pulp chamber and remove all composite base material and gutta percha 2-3mm apical to the cemento - enamel junction.
 - Place glass ionomer base 1mm thick to ensure a barrier between the "sealed" root canal and whitening gel.
 - Place and light cure a thick layer of Gingival Barrier at the gingiva region of the isolated tooth.
 - Mixing of whitening gel: Using the supplied empty clear pot (with black lid) as a mixing well, dispense the required amount of powder and liquid. For two applications on one tooth, it is recommended to dispense 1 scoop of powder and 5 drops of liquid. Please note that this is only a recommendations, and that more or less liquid can be added to obtain a desired gel consistency. Immediately mix using a brush applicator until gel is homogeneous. Replace jar/bottle caps tightly.
 - Place the gel into the opened pulp chamber and onto the labial surface.
 - Leave gel on for 8 minutes. (Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations).
 - Suction off using a surgical aspirator tip.
 - Complete steps 7-9 until the desired shade is reached. Note: gel must be used within 20 minutes of mixing.
 - Wash and apply suction after the last application.
 - Remove Gingival Barrier and re-seal tooth with appropriate restorative.
- Note: Recall patients periodically for radiographic and clinical examination.

COMPOSITION

- Pola Office Liquid
 - 35% Hydrogen peroxide
 - 65% Water
- Pola Office Powder
 - 73.26% Thickeners
 - 26.2% Catalysts
 - 0.04 % Dye
 - 0.5% Desensitizing agents
- Gingival Barrier
 - 83.95% Methacrylic ester
 - 16% Silica
 - 0.04% Pigment
 - 0.01% Butylated hydroxy toluene

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