



# SHADE MODIFICATION

COLOR MODIFICATION TINTS AND MASKING AGENTS

LIFE LIKE,  
NATURAL TOOTH  
SIMULATION



## shade modification

COLOR MODIFICATION TINTS AND MASKING AGENTS

FARBCHARAKTERISIERUNGS - UND OPAQUERMATERIAL

AGENTES MASCARADORES E MODIFICADORES DE COR

TINTES MODIFICADORES DE COLOR Y AGENTES ENMASCARADORES

AGENTS MASQUANT ET MODIFICATEUR DE TEINTES

MODIFICATORE DI TINTE PER CARATTERIZZAZIONE E OPACIZZANTI

KLEUR MODIFICATIE TINTEN EN MASKERENDE KLEUREN

TINTS TIL FARVEMODIFICERING OG MASKERING

ТИНТЕ ТИЛ ЁВУАЕМОДИФИЦЕВИНГ ОУ МАСКЕВИНГ

КРЕУВ МОДИФИКАЦИЕ ТИНТЕ ИИ МАСКЕНЕНДЕ КРЕУВЕИ

MODIFICATORE DI TINTE PER CARATTERIZZAZIONE E OPACIZZANTI

УВЕНТЕЗ МАСКУАВАНТИ ЕТ МОДИФИКАТЕУР ДЕ ТЕИНТЕЗ

ТИНТЕЗ МОДИФИЦУДОБЕЗ ДЕ СОГОВ А УВЕНТЕЗ ЕИИУВЗСВУДОБЕЗ

УВЕНТЕЗ МАСКУАВУДОБЕЗ Е МОДИФИЦУДОБЕЗ ДЕ СОУ

ЉУВРШУВРКЛЕИВИЕНУНОЗ - ПНО ОВУОПЕНУВЛЕИВУ

СОГОУ МОДИФИЦАЦИОН ТИНТЕ УИВ МАСКИНГ УВЕНТЕЗ



MODIFICATION TINTS AND MASKING AGENTS

# COLOUR MODIFICATION TINTS AND MASKING AGENTS.


## SDI'S TINTS AND MASKING AGENTS SIMULATE A LIFE LIKE, NATURAL TOOTH

The natural tooth color is derived from the color saturated dentin in conjunction with light penetrating the translucent enamel and undergoing reflection, transmission and absorption

The SDI Shade Modification system simply and quickly achieves optimum esthetics with paint on, single component, light cured liquid Tints and Masking Agents.

## TOOTH SHADES

The selection of the appropriate Masking Agent, Tint and restorative material is controlled by the multichromatic nature of the tooth which consists of four color zones:

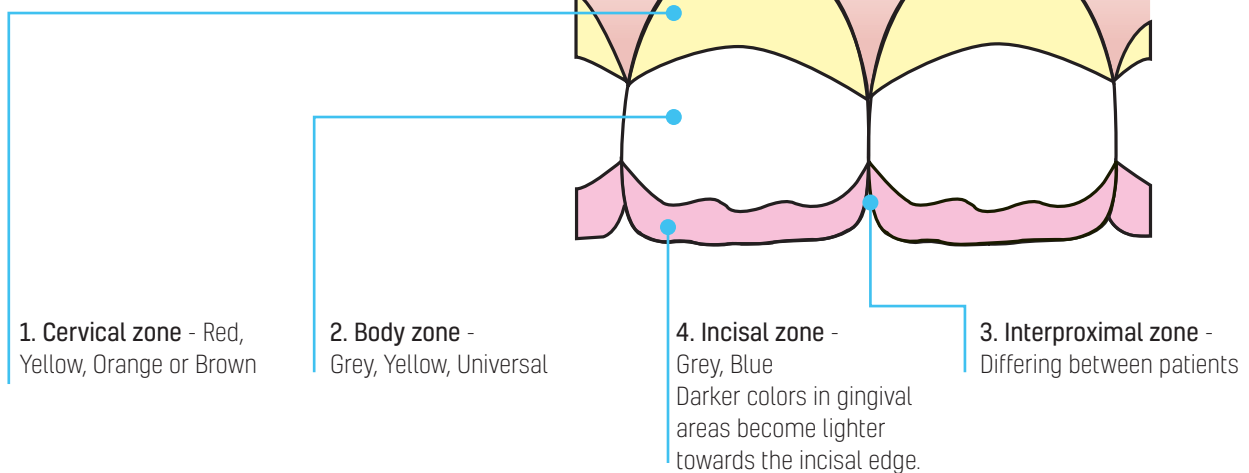

### LOW VISCOSITY

The ideal low viscosity allows for quick, easy and direct paint on application. The low viscosity resin eliminates the appearance of brush strokes and micron thin layers permit fine veneers.

### DIRECT APPLICATION

The SDI Shade Modification system's syringe and bent tip delivery system facilitates controlled direct extrusion for faster procedures.

### MASKING AGENTS

SDI Masking Agents hide enamel anomalies and discolorations to achieve natural looking restorations.

# LIFE LIKE RESTORATIONS

## MULTIPLE USES

Nine Tints and four Masking Agents ensure life like restorations are possible for even the most difficult restorative situations.

## TINTS

The chart is a resource guide for color choice. Mix shades using the layer technique to create customized color options.

<b>BLUE OR BLUE + GREY</b>	Incisal translucency Reduce brightness Cover fluorosis Hide hypocalcification
<b>BLUE</b>	Cover tetracycline stains Simulate incisal translucency
<b>YELLOW</b>	Cervical darkening Decrease brightness or increase intensity Cover bases
<b>BROWN + YELLOW</b>	Dark cervical areas Gingival highlights
<b>BROWN</b>	Cracks Tobacco stains Dark grooves Check line Decrease brightness or increase color intensity
<b>GREY</b>	Reduce intensity of other shades Incisal translucency Tobacco stains
<b>RED</b>	Create gingival High smile line Neutralize green To eliminate bluish / grey of teeth stained by tetracycline
<b>BLUE + RED</b>	Cover tetracycline stains
<b>ORANGE</b>	Cervical darkening Create cracks and dark stains Decrease brightness or increase color intensity

## INDICATIONS


Enamel
Dentin
Composite
Compomer
Porcelain
Etched glass ionomer
Metal
Carbon posts



# INSTRUCTIONS

- 1 Clean and isolate tooth
- 2 Etch tooth surface with Super Etch 37% phosphoric acid for 20 seconds
- 3 Wash thoroughly
- 4 Remove excess water. Keep moist
- 5 Apply Stae dentin/enamel adhesive to saturate all internal surfaces, or other bonding agent according to manufacturer's instructions
- 6 Apply appropriate Masking Agent or Tint in layers less than 0.5mm
- 7 Light cure for 20 seconds. Additional Masking Agent or Tint layers may be applied
- 8 Cover with a thin translucent layer of Ice Incisal, Wave, or other restorative material, according to manufacturer's instructions

# ORDER DETAILS

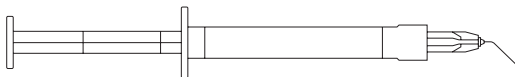

<b>Shade Modification Kit</b>	7900000
9 x 1g Tints: Blue, Yellow, Clear, Red, White, Orange, Grey, Brown, Green	
4 x 1g Masking Agents: Grey, Universal, White, Dark Yellow	
Accessories	
<b>Masking Agent 1g Refill</b>	
Universal	7970202
Dark Yellow	7970404
Grey	7975151
White	7975252
<b>Shade Tint 1g Refill</b>	
Yellow	7981414
Grey	7982424
Brown	7983333
White	7985252
Blue	7985353
Clear	7985454
Green	7985555
Orange	7985656
Red	7985757



YOUR SMILE. OUR VISION.



11-2019

M790001 C



+D036M7900011+

MADE IN AUSTRALIA  
by SDI Limited  
Bayswater, Victoria 3153  
Australia 1800 337 003  
[www.sdi.com.au](http://www.sdi.com.au)

**AUSTRIA** 00800 0225 5734  
**BRAZIL** 0800 770 1735  
**FRANCE** 00800 0225 5734  
**GERMANY** 0800 100 5759  
**ITALY** 00800 0225 5734

**NEW ZEALAND** 0800 734 034  
**SPAIN** 00800 0225 5734  
**UNITED KINGDOM** 00800 0225 5734  
**USA & CANADA** 1 800 228 5166