SDI AURA BULK FILL

UNIVERSAL BULK FILL COMPOSITE

INSTRUCTIONS FOR USE

Made in Australia by SDI Limited
3-15 Brunston Street
Bayswater, Victoria 3153
Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
United Kingdom 0800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au

EC REP

SDI Germany GmbH
Hansestrasse 85
51149 Cologne
Germany
T+49 2203 9255 0

PA41332.C

SDI AURA BULK FILL

ULTRA UNIVERSAL
RESTORATIVE MATERIAL
INSTRUCTIONS FOR USE

ENGLISH

INTRODUCTION

aura bulk fill is a light cured, radiopaque, high strength, composite designed to eliminate time consuming layering for posterior restoration. The **aura bulk fill** has a 5mm depth of cure allowing fast and easy placement when restoring deep cavities.

INDICATIONS

- Posterior restorations
- Core buildup
- Splinting
- Sandwich technique with glass-ionomer

CONTRA-INDICATIONS

- Pulp capping.
- Do not use in conjunction with any eugenol containing materials.
- Anyone with allergy to acrylics

PLACEMENT PROCEDURE:

1. Isolate tooth.
2. Prepare cavity using standard technique for composite resin restorations.
Note: In cases where pulp protection is necessary, use a calcium hydroxide liner.
3. Apply a dentin/enamel bonding agent such as SDI Zipbond Universal as per manufacturer's instructions.
4. Place **aura bulk fill** no more than 5mm thick.
5. Light cure for 20 seconds using a high powered LED curing light (460 - 480nm wavelength).
6. Finish restoration using standard technique.

STORAGE AND HANDLING

- Store at temperatures between 10° and 25°C (50°-77°F).
- Use at room temperature between 20° and 25°C (68°- 77°F).
- Syringe system: when not in use replace the cap tightly.

PRECAUTIONS

- Avoid prolonged contact of **aura bulk fill** with the skin or oral tissue, as it may cause inflammation of the oral tissues or skin sensitization.
- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Do not expose material to direct light.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID

- **Eye (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol. Wash thoroughly with water.
- **Ingestion:** Rinse mouth thoroughly. Drink plenty of water/milk. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.

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