SDI POLA WHITENING STAND


ENGLISH

INSTRUCTIONS FOR USE

Thank you for purchasing the Pola whitening stand for use with the Radii Xpert and Radii Plus curing lights from SDI. For operation and safety for the Radii Xpert or Radii Plus LED curing light, please refer to the instructions supplied with those products. This instruction sheet only details mounting operations for the Pola whitening stand purchased.





Within this kit you will find a mount support for both the Radii Xpert and the Radii Plus curing lights. Please choose the appropriate mount support depending on the curing light you have purchased.

Radii Plus mount support Radii Xpert mount support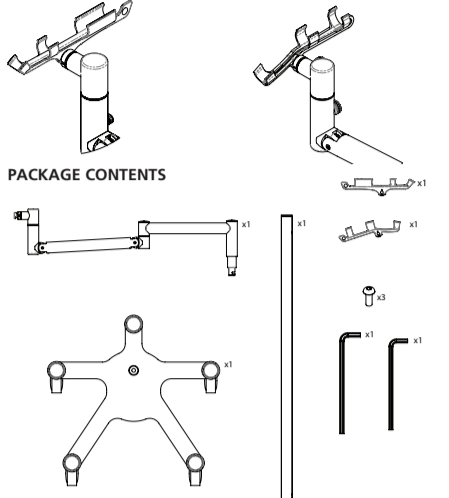

- The box contains:**
- Trolley base (x1)
 - Trolley arm (x1)
 - Trolley pole (x1)
 - Mount support for Radii Plus curing light (x1)
 - Mount support for Radii Xpert curing light (x1)
 - Rollstativ-Fuß (x1)
 - Rollstativ-Arm (x1)
 - Rollstativ-Stange (x1)
 - Halterung für Polymerisationslampe Radii Plus (x1)
 - Halterung für Polymerisationslampe Radii Xpert (x1)
 - Schrauben (x3)
 - Allen key 2,5 (x1)
 - Allen key 2 (x1)
 - Instructions manual

MOUNTING

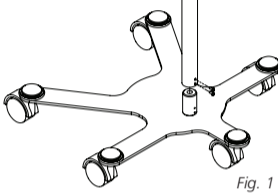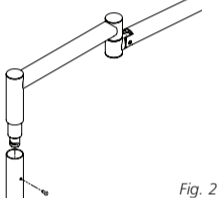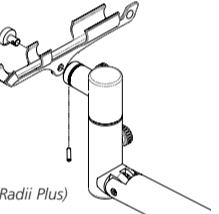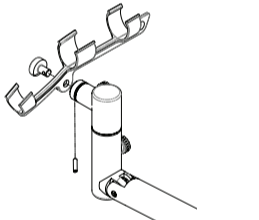- Mount the pole on the trolley as indicated in figure 1 and keep it in position using two (2) screws. Align the holes between the pole and the base correctly. 
- Mount the arm as indicated in Fig. 2 and keep it in position with the remaining screw. 
- Mount the desired support (for the Radii Plus or the Radii Xpert) and secure it in position using the socket set screw shown in figure 3 (Radii Plus) and figure 4 (Radii Xpert). And regulate the smoothness of the movement using the 2 knobs. 

CLEANING THE WHITENING STAND

Please use only water based cleaning agents. Avoid the use of abrasive materials and cleaning substances containing alcohol, trichloroethylene, petrol, turpentine or similar ingredients.

SAFETY AND PRECAUTIONS

No alterations or modifications should be made to this appliance. Please ensure that trolley, pole, arm and light support are firmly connected.

WARRANTY PERIOD

SDI Ltd, the Manufacturer, extends the following warranty periods:

- Two (2) year warranty for the Pola whitening stand. Warranty covers defects in materials or workmanship to the original purchaser of this product. The Manufacturer agrees to correct any defects which develop within the warranty period, either by repair or replacement, at its option. This warranty is valid providing factory inspection indicates that any such defect developed during normal and proper use subject to the conditions below.

Warranty conditions – please read carefully

Go to www.sdi.com.au/warranty to register your warranty.

- Claims for damage in shipment should be filed promptly with the transportation company.
- All shipments claimed defective can only be returned to the Manufacturer with the written consent of the Manufacturer. All returned products must be accompanied by a full description of discrepancy or malfunction.
- Use only SDI Ltd parts exclusively to replace defective components. The product warranty does not cover any damage resulting from the use of third-party replacement parts.
- Manufacturer shall be released from all obligations under this warranty in the event of improper installation; damage due to casualty; or obvious misuse including but not limited to damage from liquids.
- Manufacturer shall in no event be liable for any consequential damages arising from the misuse of the product.
- Only the above stated warranty shall be applicable, and all other warranties, expressed or implied, are hereby excluded. All correspondence concerning the product should specify the model number.

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SDI Ltd, de producent, hanteert de volgende garantieperioden: