SDI RIVA COAT

LIGHT CURED COATING MATERIAL
FOR GLASS IONOMER PRODUCTS
INSTRUCTIONS FOR USE

E
N
G
L
I
S
H

Riva Coat is designed for the prevention of saliva and water contamination to exposed surfaces of glass ionomer cements during the first 24 hours immediately after placement.

INDICATIONS

- Glass ionomer cement restorations requiring a coating material after placement.

CONTRAINDICATIONS

- Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

INSTRUCTIONS FOR USE

1. Place the glass ionomer cement according to manufacturer's instructions.
2. When restoration has hardened, apply Riva Coat to all exposed surfaces of restoration.
3. Light cure for 20 seconds.
4. Replace cap immediately after use.

STORAGE AND HANDLING

- Material is light sensitive. Do not dispense Riva Coat until ready to use.
- Store in a dark place at temperatures between 2° and 25°C (35° - 77°F). Use at room temperature between 20° and 25°C (68° - 77°F).

PRECAUTIONS

- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID

- **Eye (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol / acetone. Wash thoroughly with water.
- **Ingestion:** Drink plenty of water. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.

PRODUCT RANGE

- 5mL bottle refill

Made in Australia by SDI Limited
3-15 Brunson Street
Bayswater, Victoria 3153
Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
New Zealand 0800 734 034
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au

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