

SCULPTING AND
MODELLING AGENT

INSTRUCTIONS FOR USE

Dipping Resin from SDI Limited is a HEMA free, BPA free, colourless, low viscosity, unfilled light cured resin. It is designed to eliminate uncured composite sticking to instruments, making it easier to manipulate and sculpt.

COMPOSITION:

- Unfilled urethane dimethacrylate resin.

INDICATIONS:

- Dipping resin for composite instruments for sculpting and manipulation of composite restorations.

CONTRAINDICATIONS:

- Bonding of composite to dentine.
- People with allergies to acrylic resins.

PROCEDURE:

1. Dispense 1-2 drops of Dipping Resin into a mixing well and replace cap onto the bottle.
2. Place initial increment of composite to appropriately prepared site.
3. Dip composite instrument into Dipping Resin, sculpt and manipulate the composite to desired shape.
4. Light cure according to manufacturer's instructions and finish restoration using standard polishing techniques.

STORAGE AND HANDLING:

- Dipping resin is light sensitive, do not dispense until ready to use.
- Store at temperatures between 10° and 25°C (50° - 77°F).

PRECAUTIONS:

- For professional use only.
- Keep away from sources of direct sunlight.
- Keep out of reach of children. Do not take internally.
- Do not use after expiry date.
- Avoid prolonged contact of uncured Dipping Resin with the skin or oral tissue, as it may cause inflammation of the oral tissues or skin sensitization.
- Avoid contact with eyes.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID:

- **Eye (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol / acetone. Wash thoroughly with water.
- **Ingestion (large amount):** Drink plenty of water. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.

PACKAGING:

- Pack of 1 x 5mL bottle



Made in Australia by SDI Limited
3-15 Brunsdon Street
Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au

PAK41306 C

