**POLA OFFICE+**  
ADVANCED TOOTH  
WHITENING SYSTEM

E  
N  
G  
L  
I  
S  
H

**INSTRUCTIONS FOR USE**

Pola Office+ is a hydrogen peroxide based in-office tooth whitening system requiring minimal chair time. It is a neutral pH gel and contains desensitizers to maximize patient comfort. The dual barrel syringe system mixes as the gel is applied directly to the tooth.

- INDICATIONS:**
- Whitening of discoloured vital teeth.
  - Whitening of discoloured non-vital teeth.

**NET WEIGHT:**  
Depending on kit purchased, 3.3g syringes. Please refer to outside of box.

- CONTRA-INDICATIONS:**
- Do not use on pregnant or lactating women.
  - Do not use on children under 14 years of age.
  - Do not use the Gingival Barrier on any persons having known resin allergies.
  - Do not use Pola Office+ on any persons having known peroxides allergies.
  - Pola Office+ will not lighten any restorative materials.
  - Do not use on patients with extremely sensitive teeth.

- PRECAUTIONS:**
- For professional use only.
  - Do not remove syringe from wrapper until ready to use - Pola Office+ is heat and light sensitive.
  - Keep out of reach of children.
  - SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.
  - Single use only - do not recap un-used material in syringe, as contamination of the two gels will render Pola Office+ unstable.
  - Pola Office+ syringes may be under pressure - remove cap and attach nozzle with caution.
  - Patient must wear protective eyewear.
  - Operators must wear gloves, mask and protective eyewear.
  - Anyone with a history of chemical allergies is advised to undergo allergy checks before treatment.
  - Do NOT anaesthetize patient.
  - Do not allow Pola Office+ gel to come in contact with skin, eyes and soft tissues.

- FIRST AID:**
- For advice, contact a doctor or a Poisons Information Centre, at once. Seek urgent hospital treatment if needed.
  - SKIN/TISSUES (contact):** Immediately wash thoroughly with water. Apply a slurry of sodium bicarbonate and water to the affected area. Seek medical attention if symptom persists.
  - EYES (contact):** Open the eye wide and thoroughly wash for 15 minutes with running water. Seek medical attention.
  - INGESTION:** If swallowed, do NOT induce vomiting. Rinse mouth with water. Gargle with salt water and drink lots of milk or water. Seek medical attention if symptoms persist.
  - INHALATION:** no symptoms expected.

**STORAGE AND HANDLING:**  
Store in a cool place (2°-25°C / 35°-77°F). For best results, keep refrigerated.

- TIPS:**
- Select appropriate candidate. Note: Not everyone is suitable for in-office whitening e.g. patient with translucent teeth or in the Vita® C group.
  - Patient must have good overall health.
  - Seal all exposed roots and restorations.
  - Inform patient that results are not guaranteed as the removal of stains varies between patients.
  - Acid etching before treatment is NOT recommended.
  - Large diameter suction tips are not recommended as they can spread the gel to other areas.
  - Mild discomfort in teeth during treatment can be a common side effect.
  - If patient experiences sensitivity during light curing, move the tip slightly away from the tooth. If this doesn't work, remove gel and rinse the affected area. Assess other possible causes.
  - Delay any esthetic restorations for 2 to 4 weeks after treatment.
  - Advise patient not to smoke, consume highly coloured, hot or cold foods or beverages for at least 2 days.
  - Treat any post-operative sensitivity with a desensitising gel or toothpaste.
  - Prescribe Pola Day or Pola Night to maintain brightness if desired.
  - A patient consent form is available for download at [www.sdi.com.au](http://www.sdi.com.au)

- INSTRUCTIONS FOR VITAL TOOTH WHITENING:**
- Determine and record pre-operative shade. Please follow this Vita® shade guide arrangement according to degree of brightness: **B1•A1•B2•D2•A2•C1•C2•D4•A3•D3•B3•A3.5•B4•C3•A4•C4**
  - Clean teeth with ONLY a flour based pumice.
  - Place cheek retractors and then cover exposed lip surface with petroleum gel.
  - Dry teeth and apply Gingival Barrier to both arches, slightly overlapping enamel and interproximal spaces.
  - Light cure in a fanning motion for 10-20 seconds until Gingival Barrier is cured.
  - Firmly** attach a mixing nozzle to the Pola Office+ syringe away from patient. Dispense a small amount of gel on to a mixing pad until a uniform gel is extruded.
  - Using the nozzle as a guide, directly apply a **thin** layer of gel to all teeth undergoing treatment. A thin layer will help prevent the gel from running. NOTE: It is best not to lift the nozzle when moving from tooth to tooth. Use a continuous motion.
  - Leave gel on for 8 minutes (Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations). Remove gel off using a surgical aspirator tip. Note: It is normal for a residue layer of gel to remain on the surface of the teeth.
  - Repeat steps 7-9 three times (optional 4 times).
  - After the last application, suction all the gel off, then wash and apply suction.
  - Remove Gingival Barrier by lifting it from one end.
  - Record the final shade.

- TIPS FOR NON-VITAL TOOTH WHITENING:**
- Isolate tooth effectively.
  - Protect oral mucosa to prevent chemical burns.
  - Verify adequate endodontic obturation.
  - Use protective barrier to prevent peroxide from reaching the periodontal ligament.
  - Avoid acid etching as periodontal ligament irritation may result.
  - Advise the patient that a whitened non-vital tooth will not look the same as a vital tooth.
  - Recall patients periodically to examine the whitened teeth (clinically & radiographically).

- INSTRUCTIONS FOR NON-VITAL TOOTH WHITENING:**
- Determine and record pre-operative shade. Please follow this Vita® shade guide arrangement according to degree of brightness: **B1•A1•B2•D2•A2•C1•C2•D4•A3•D3•B3•A3.5•B4•C3•A4•C4**
  - Isolate tooth/teeth undergoing whitening using a rubber dam. Ensure the dam fits tightly at the cervical margin. Note: If a rubber dam is not used, cover approximately half of the adjacent vital teeth with 3-4mm thick layer of Gingival Barrier to insulate from heat discomfort of the curing light. This is done when one tooth is being whitened non-vitaly.
  - Open lingual access to pulp chamber and remove all composite, base material and gutta percha 2-3 mm apical to the CE junction. Place a glass ionomer/zinc phosphate base, at least 1 mm thick to ensure a barrier between the "sealed" root canal and whitening gel.
  - Place the Pola Office+ gel into the opened pulp chamber and onto the labial surface.
  - Leave gel on/in for 8 min (Optional: curing light may be used at this stage, please refer to light manufacturer's recommendations) NOTE: If using a curing light and treating inner and outer surfaces, 2 curing lights can be used (one on each side of tooth) to activate Pola Office+.
  - Suction off the gel.
  - Repeat steps 4-6 until desired shade is achieved.
  - After the last application, suction and wash the gel off.
  - Record the final shade.

\*Vita shade guide is a trademark of VITA Zahnfabrik, H. Rauter GmbH & Co. KG, D-79713 Säckingen, Germany

**PACKAGING:**  
**3 Patient Kit**  
3 x 2.8ml Pola Office+ syringes  
3 x 1g Gingival Barrier syringes accessories

**1 Patient Kit**  
1 x 2.8ml Pola Office+ syringes  
1 x 1g Gingival Barrier syringe accessories

**Bulk Kit**  
10 x 2.8ml Pola Office+ syringes accessories

Made in Australia by SDI Limited  
3-15 Brunsdon Street  
Bayswater, Victoria 3153  
Australia

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