# SDI pola luminante

Brush-on tooth whitening system  
(6% Hydrogen Peroxide)  
INSTRUCTIONS FOR USE

ENGLISH

**Pola Luminante** is a pleasantly flavoured 6% hydrogen peroxide whitening solution. Designed to brush directly on teeth to form a thin whitening film without the need for trays. **Pola Luminante** easily fits into your pocket or handbag for use at your convenience. **Pola Luminante** is a great way to maintain your white smile anywhere and anytime. Before using **Pola Luminante**, your dental practitioner should assess your suitability for the product including your medical history and explain the application procedure.

**INDICATIONS FOR USE:**  
Whitening of discoloured teeth

- INSTRUCTIONS:**
1. Blot dry the front surface of teeth with a tissue.
  2. Whilst retracting the lips, apply a thin layer of Pola Luminante solution to the front surfaces of the teeth, re-loading brush as required.
  3. After application do not let your lips touch your teeth for 30 seconds to ensure a thin whitening film forms on the teeth.
  4. Leave Pola Luminante solution on teeth for a minimum of 30 minutes.
  5. Do not eat or drink during this time to maximize results.
  6. Use Pola Luminante twice a day for 2 weeks for best results.
  7. Always follow the advice of your dental practitioner.

For best results, one of the daily treatments should be applied before going to bed, allowing the **Pola Luminante** to stay on overnight. When not in use, **firmly** replace cap to prevent solution from drying out.

**STORAGE AND HANDLING:**

- Store in a cool place, out of direct sunlight at temperatures between (2°-25°C / 35°-77°F).

**PRECAUTIONS:**

- Some users may experience sensitivity or irritation. If this occurs, it is usually mild and transient. If symptoms persist, seek advice from your dental practitioner.
- Irritant. Avoid contact with eyes or gums. Avoid swallowing.
- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Not to be used by pregnant and lactating women.
- Avoid contact with eyes, rinse immediately if the product comes into contact with them.
- The countries listed below have age restrictions for whitening products, where it is not to be used under a certain age: All European countries: under 18 years of age; New Zealand: under 16 years of age; Canada: under 12 years of age. Note: For age restrictions of other countries, refer to your dental association.
- Do not apply to damaged, inflamed or compromised skin.
- Use at room temperature.
- Pola Luminante should not be used by patients with known allergies or chemical sensitivity to peroxides or glycerol.
- Do not smoke during or immediately after the whitening process.
- Keep product away from heat and naked flame, as product is flammable.
- Discard Pola Luminante after treatment has been completed.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.
- Contains sodium fluoride and contains hydrogen peroxide.
- Do not use after expiry date.
- Keep out of reach of children and pets.


**FIRST AID:**  
For advice, contact (a Poisons Information Centre) or a doctor.

- **Eye (contact):** Hold eyelids open and flush with running water for 10-15 minutes. Seek medical advice if symptoms persist.
- **Skin (contact):** Remove contaminated clothing and flush skin with running water. Wash gently with soap and water, and rinse well.
- **Ingestion:** If swallowed, do NOT induce vomiting. Call the Poisons Information Centre or seek medical advice.
- **Inhalation:** No symptoms expected.

Made in Australia by SDI Limited  
Bayswater, Victoria 3103  
Australia 1 800 337 003  
Austria 00800 0225 5734  
Brazil 00800 770 1735  
France 00800 0225 5734  
Germany 0800 1005759  
Ireland 00800 0225 5734  
Italy 00800 0225 5734  
New Zealand 0800 734 034  
Spain 00800 0225 5734  
United Kingdom 0800 0225 5734  
USA & Canada 1 800 228 5166

[www.sdi.com.au](http://www.sdi.com.au)  
[www.polawhite.com.au](http://www.polawhite.com.au)

EC REP  
SDI Germany GmbH  
Hansestrasse 85  
51149 Cologne  
Germany  
T+49 2203 9255 0

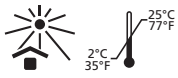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