SDI

POLA RETRACTORS

INSTRUCTIONS FOR USE

E
N
G
L
I
S
H

These convenient and practical cheek retractors are designed to keep cheeks and lips retracted in order to efficiently carry out teeth whitening procedures with Pola products from SDI. They conveniently hold back the tongue, and allow the patient to bite down posteriorly.

PROCEDURE

1. Place the cheek retractor with the bracket facing towards the patient's tongue.
2. The plastic lip in the middle of the bracket must face down towards the tongue (See photo).
3. The patient can bite the bracket during treatment.

CLEANING INSTRUCTIONS

1. The cheek retractor is recommended as a single use product.
2. The cheek retractor can be disinfected after use.
3. Disinfect before each use.

4. DO NOT AUTOCLAVE.

CAUTION

1. Before each use, the cheek retractor must be checked for damage and tested by bending the crescent-shaped cheek holders to ensure that there is no risk of breakage.
2. Please store the retractors at room temperature in a dry place away from light.
3. Shelf life with proper use 2 years from date of delivery.

Made in Germany by:
Prodent Dentalbedarf GmbH
Nicolaus-Zech-Str. 36
96450 Coburg

EC REP

Prodent Dentalbedarf GmbH
Nicolaus-Zech-Str. 36
96450 Coburg, Germany

Distributed by:
SDI Germany GmbH
Hansestrasse 85
51149 Cologne,
Germany
T+49 2203 9255 0
www.sdi.com.au

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