SDI **RIVA STAR**
INSTRUCTIONS FOR USE
(BOTTLE)
 Riva Star is a new generation desensitiser that immediately relieves tooth sensitivity.
INDICATIONS FOR USE:
 Desensitising cervical tooth hypersensitivity.
CONTRA-INDICATIONS
 Do not use on patients undergoing thyroid gland therapy, who have a history of chemical allergies such as silver, fluoride, potassium, iodine and ammonia. **Note:** If in doubt, consult a doctor or an allergy specialist before use.
PROCEDURE:
 1. Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
 2. Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
 3. Dry area to be treated and isolate it using rubber dam or a light curable Gingival Barrier (SDI Limited).
 4. Protect any exposed gingival tissues and lips with petroleum jelly/ cocoa butter taking care not to contaminate treatment site. When not using rubber dam, isolate area with cotton rolls.
 5. **Riva Star Step 1:** Dispense one (1) drop of the solution (**grey** label) on a non absorbent mixing pad or dappen dish.
 6. Carefully apply solution to treatment site only, a medium sized micro brush, such as Points (SDI Limited) is recommended.
 7. **Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (**green** label) onto fresh non-absorbent mixing pad.
 8. Apply a generous amount of the solution to treatment site, a medium sized micro brush, such as Points (SDI Limited) is recommended.
 9. Apply same solution to any other tooth surfaces that may have come into contact with the solution from the grey bottle. **Note:** Treatment surface initially appears creamy white, keep applying Riva Star Step 2 solution (**green** label) until it turns clear.
 10. Wash thoroughly with water for at least 10 seconds and air dry. **Note:** Do not desiccate surface if a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited), is to be used.
 11. Remove all protective/isolation materials used in the mouth.

**E
N
G
L
I
S
H**

IMPORTANT: Ensure that both Step 1 and Step 2 bottle surfaces (Eg tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

11. Clean dappen dishes and discard used mixing pads and micro-brushes in accordance with local regulations.

DOSAGE:
 • One (1) drop of Riva Star Step 1 and two (2) drops of Riva Star Step 2 can treat up to 5 teeth per patient.
 • Limit of one (1) drop of Riva Star Step 1 and two (2) drops of Riva Star Step 2 drops per visit. If needed a second application of Riva Star may be administered after 1 week.

PRECAUTIONS:
 • For professional use only.
 • Do not repeat treatment on pregnant or lactating women.
 • Patient must be informed of potential staining prior to treatment.
 • Due to potential staining, it is recommended that anterior teeth not be treated.
 • Avoid contact of solution from Riva Star Step 1 (**grey** label) with oral soft tissue, eyes, skin and clothing. If accidental contact occurs, refer to "FIRST AID" below.
 • Avoid contact of solution from Riva Star Step 2 bottle (**green** label) with eyes.
 • Do not desensitize teeth with soft dentine as staining may occur.
 • **Note:** If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement (such as Riva Self-Cure, SDI Limited).
 • Keep out of reach of children.
 • Do not ingest.
 • Riva Star Step 1 solution (**grey** label) will stain (see "HANDLING STAINS").
 • Wear gloves and eye protection when handling.
 • Do not use after expiry date.
 • Caution: Federal Law restricts this device to sale by or on the order of a dentist.
 • Solution from Riva Star Step 1 (**grey** label) contains 0.06g / mL fluoride ions.
 • SDS is available at www.sdi.com.au or contact your SDI representative.

STORAGE AND HANDLING:
 • Store bottles upright in the packaging provided.
 • Store between 20°and 25°C (68°- 77°F) and away at room temperature.
 • Do not refrigerate products.
 • Do not dispense solution from **grey** label bottle or **green** label bottle until ready to use.
 • Replace cap on bottle and seal tightly immediately after use.

IMPORTANT: Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

FIRST AID:
 • **Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre (Australia, 131 126) or a doctor, or for at least 15 minutes. Seek for medical attention.
 • **Skin/oral soft tissue (contact):** Remove contaminated clothing. Flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
 • **Ingestion:** If affected, do NOT induce vomiting. Give a glass of milk or water. For advice contact a Poisons Information Center or a doctor.
 • **Inhalation:** If affected, remove from contaminated area.

HANDLING STAINS:
 • **Skin:** In case of skin contact with Riva Star Step 1 (grey label), apply Riva Star Step 2 (green label) with gentle rubbing. Stain will slowly disappear. Otherwise, if Step 2 is not applied, the stain should disappear gradually after a few days.
 • **Clothing:** Remove contaminated clothing and soak in Napisan® (or other fabric stain remover) overnight and wash per normal.
 • **Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan® is not the registered trademark of SDI Limited.

PACKAGING:
 Each kit contains:
 - 1 x Riva Star Step 1 (**grey** label) bottle 1.5 mL
 - 1 x Riva Star Step 2 (**green** label) bottle 3 mL

| | |
|--|---|
| <p>US Patents granted.</p> <p>Made in Australia by SDI Limited 3-15 Brunson Drive Bayswater, Victoria 3153 Australia</p> | <p>EG TREP</p> <p>SDI Germany GmbH Hansestrasse 85 51149 Cologne Germany T+49 2203 9255 0</p> |
| <p>Australia 1 800 337 003 Austria 00800 0225 5734 Brazil 0800 770 1735 France 00800 0225 5734 Germany 0800 1005759 Italy 00800 0225 5734 Spain 00800 0225 5734 United Kingdom 00800 0225 5734 USA & Canada 1 800 228 5166 www.sdi.com.au www.rivastar.com</p> | <p>CE 0297</p> <p>PK41361-D</p> |

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