



acid etch gel acid etch liquid

E
N
G
L
I
S
H

INSTRUCTIONS FOR USE

acid etch gel and acid etch liquid are 37% phosphoric acid etchant for etching enamel and dentin.

Indications

1. Etching of enamel
2. Etching of dentine

Contraindications

1. Soft tissues
2. If the treatment site cannot be etched
3. Patients who are allergic to phosphoric acids and blue dyes

INSTRUCTIONS:

1. Isolate tooth. Prepare cavity.
2. Clean the surface to be etched using a non-fluoridated, oil-free prophylaxis paste, or a pumice and water slurry.
3. Rinse thoroughly with water.
4. Thoroughly dry the surface to be etched with dry, oil-free air.
6. Carefully dispense required amount of **acid etch gel / acid etch liquid** into a dappen dish.
7. Use a microbrush or brush to apply the gel/liquid to etching site.
8. Leave on:
 - a. enamel for at least 30 seconds.
 - b. dentin and glass ionomer liner for 20 seconds.
9. Rinse well with water for 5-10 seconds.
5. Remove excess water.
6. Apply bonding agent, such as **Stae**, or pit and fissure sealant, such as **Conseal f**, according to manufacturer's instructions.

PRECAUTIONS:

- Corrosive - Avoid contact with oral tissue, eyes and skin. If accidental contact occurs wash with copious amounts of water. In case of eye contact, wash eye for 15 minutes and seek medical attention.
- Keep out of reach of children.
- Do not take internally.
- When not in use replace the bottle cap tightly.
- Use at room temperature.
- Do not use after expiry date.
- When not in use keep the cap tightly sealed.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID:

- Eye (contact): Wash thoroughly with water and seek urgent medical attention.
- Skin (contact): Wash thoroughly with water. Seek medical attention.
- Ingestion: Do NOT induce vomiting. Rinse mouth. Drink plenty of water/milk. Seek medical attention.
- Inhalation: No symptoms expected.

 Made in Australia by SDI Limited
Bayswater, Victoria 3153
Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Ireland 00800 0225 5734
Italy 00800 0225 5734
New Zealand 0800 734 034
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au
www.polawhite.com.au

EC	REP
----	-----

SDI Germany GmbH
Hansestrasse 85
51149 Cologne
Germany
T+49 2203 9255 0

