
# poloffice bulk kit

ADVANCED TOOTH WHITENING SYSTEM

Made in Australia by SDI Limited  
 Bayswater, Victoria 3153  
 Australia 1 800 337 003  
 Austria 00800 0225 5734  
 Brazil 0800 770 1735  
 France 00800 0225 5734  
 Germany 0800 1005759  
 Ireland 00800 0225 5734  
 Italy 00800 0225 5734  
 New Zealand 0800 734 034  
 Spain 00800 0225 5734  
 United Kingdom 00800 0225 5734  
 USA & Canada 1 800 228 5166  
 www.sdi.com.au  
 www.polawhite.com.au

PAK1317 B

Last Revised: 2016-10

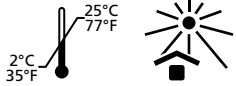1. Determine and record pre-operative shade.
2. Please follow this Vita® shade guide arrangement according to degree of brightness: B1+A1+B2+D2+A2+C1+C2+D4+A3+D3+B3+A3.5+B4+C3+A4+C4
3. Clean teeth with **ONLY** a flour based pumice.
4. Place cheek retractors and then cover exposed lip surface with petroleum gel.
5. Dry teeth and apply Gingival Barrier to both arches, slightly overlapping enamel and interproximal spaces.
6. Light cure in a fanning motion for 10-20 seconds until Gingival Barrier is cured.
7. **Mixing of whitening gel:** Using the supplied empty clear pot (with black lid) as a mixing well, dispense the required amount of powder and liquid. Recommendations are listed in the below table. Please note that these are only recommendations, and that more or less liquid can be added to obtain a desired gel consistency. Replace jar/bottle caps tightly.
8. **1 tooth, 2 applications**  
1 level scoop of powder  
5±1 drops of liquid
9. **Single arch, 1 application**  
2 level scoops of powder  
10±2 drops of liquid
10. **Dual arch, 1 application**  
4 level scoops of powder  
20±3 drops of liquid
11. Immediately mix using a brush applicator until gel is homogeneous.
12. Apply a **thick** layer of gel to all teeth undergoing treatment. Leave gel on for 8 minutes (Optional: curing light may be used at this stage, please refer to light manufacturer's recommendations)
13. Suction off using a surgical aspirator tip.
14. Complete Steps 6-10 three times (optional 4 times). Note: gel must be used within 20 minutes of mixing.
15. After the last application, suction all the gel off, then wash and apply suction.
16. Remove Gingival Barrier by lifting it from one end.

- INSTRUCTIONS FOR NON-VITAL TOOTH WHITENING.**
1. Verify adequate endodontic obturation.
  2. Isolate tooth with rubber dam.
  3. Open lingual access to pulp chamber and remove all composite base material and gutter percha 2-3mm apical to the cemento - enamel junction.
  4. Place glass ionomer base 1mm thick to ensure a barrier between the "sealed" root canal and whitening gel.
  5. Place and light cure a thick layer of Gingival Barrier at the gingiva region of the isolated tooth.
  6. **Mixing of whitening gel:** Using the supplied empty clear pot (with black lid) as a mixing well, dispense the required amount of powder and liquid. For two applications on one tooth, it is recommended to dispense 1 scoop of powder and 5 drops of liquid. Please note that this is only a recommendations, and that more or less liquid can be added to obtain a desired gel consistency. Immediately mix using a brush applicator until gel is homogeneous. Replace jar/bottle caps tightly.
  7. Place the gel into the opened pulp chamber and onto the labial surface.
  8. Leave gel on for 8 minutes. (Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations).
  9. Suction off using a surgical aspirator tip.
  10. Complete steps 7-9 until the desired shade is reached. Note: gel must be used within 20 minutes of mixing.
  11. Wash and apply suction after the last application.
  12. Remove Gingival Barrier and re-seal tooth with appropriate restorative. Note: Recall patients periodically for radiographic and clinical examination.

- COMPOSITION**
- **Pola Office Liquid:** Hydrogen peroxide, Water
  - **Pola Office Powder:** Thickeners, Catalysts, Dye, Desensitizing agents
  - **Gingival Barrier:** Methacrylic ester, Silica, Pigment, Butylated hydroxy toluene

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- TIPS:**
1. Patient must have good overall health.
  2. Seal all exposed roots and restorations.
  3. Inform patient that results are not guaranteed as the removal of stains varies between patients.
  4. Acid etching before treatment is NOT recommended.
  5. Large diameter suction tips are not recommended as they can spread the gel to other areas.
  6. Mild discomfort in teeth during treatment can be a common side effect.
  7. If patient experiences sensitivity during light curing, move the tip away slightly. If this doesn't work, remove gel and rinse the affected area. Assess other possible causes.
  8. Delay any aesthetic restorations for 2 to 4 weeks after treatment.
  9. Advise patient not to smoke, consume highly coloured, hot or cold foods or beverages for at least 2 days.
  10. Treat any post-operative sensitivity with a desensitising gel or toothpaste.
  11. Prescribe Pola Day or Pola Night to maintain brightness if desired.
  12. SDS available at www.sdi.com.au or contact your regional representative.
  13. A patient consent form is available for download at www.sdi.com.au

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