3%, 7.5% and 9.5% hydrogen peroxide  
10%, 16%, and 22% carbamide peroxide  
Advanced tooth whitening system

## INSTRUCTIONS FOR USE FOR PATIENTS AND DENTAL PROFESSIONALS

Pola Day and Pola Night are high viscosity, neutral pH take home tooth whitening gels. Both are pleasantly flavoured, crystal clear sustained release gels with a unique blend of soothers, conditioners and a water content high enough to further reduce sensitivity.

### INDICATIONS FOR USE:

- Whitening of discoloured vital teeth.
- Whitening of discoloured non-vital teeth.

### COMPOSITION:

**Pola Day:** Hydrogen peroxide, Additives, Glycerol, Water, Flavour

**Pola Night:** Carbamide peroxide, Additives, Glycerol, Water, Flavour

### HYDROGEN PEROXIDE EQUIVALENT:

- Pola Night 10% Carbamide Peroxide = 3.3% Hydrogen Peroxide
- Pola Night 16% Carbamide Peroxide = 5.3% Hydrogen Peroxide
- Pola Night 22% Carbamide Peroxide = 7.3% Hydrogen Peroxide


### TREATMENT GUIDELINES:

	3%	7.5%	9.5%
Pola Day	2 or 3 x 30 minutes / day or 1 x 60 minutes / day	1 or 2 x 30 minutes / day or 1 x 45 minutes / day	2 x 15 minutes/day or 1 x 30 minutes / day
Pola Night	10%	16%	22%
	1 x 2 hours / day to overnight	1 x 90 minutes / day to overnight	1 x 45 minutes / day

Note: Patients with sensitivity will require a day or two break between treatments. Shortening treatment time is also suggested.

### PLACEMENT PROCEDURE:

- Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
- Seat the tray, with the gel around the teeth.
- Wipe away excess gel in mouth with a tissue or dry soft brush.
- After treatment, remove tray. Rinse tray and mouth with lukewarm water.
- Brush teeth.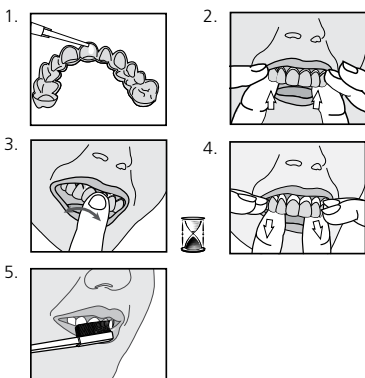

### FOR EFFECTIVE RESULTS:

- Do not eat, drink or smoke during treatment.
- Do not smoke immediately after treatment - wait for at least two hours.
- Foods and drinks containing strong colours should be avoided for at least 48 hours after the treatment period or consumed in moderation.
- Use gel at room temperature.

### STORAGE AND SHELF LIFE:

- Store in a cool and out of direct sunlight at temperatures between (2°-25°C / 35°-77°F).
- Do not use after expiration date.

### PRECAUTIONS:


- To be used under the supervision of a dentist.
- Keep out of reach of children and pets.
- Not to be used by pregnant or lactating women or children under 14 years old.
- Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using these products.
- Discontinue use if any unusual sensitivity or reactions occur and consult your dentist.
- Do not inject Pola Day or Pola Night into the body.
- CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.

### FIRST AID:

- Eye (contact):** Hold eyelids apart and flush with running water for at least 15 minutes. Seek medical advice if symptoms persist.
- Skin (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
- Ingestion (large amount):** If swallowed, do NOT induce vomiting. Rinse mouth. Drink plenty of milk or water and seek medical advice.
- Inhalation:** No symptoms expected.

Made in Australia by SDI Limited  
Bayswater, Victoria 3153  
Australia 1 800 337 003  
Austria 00800 0225 5734  
Brazil 0800 770 1735  
France 00800 0225 5734  
Germany 0800 1005759  
Ireland 00800 0225 5734

Italy 00800 0225 5734  
New Zealand 0800 734 034  
Spain 00800 0225 5734  
United Kingdom 00800 0225 5734  
USA & Canada 1 800 228 5166  
[www.sdi.com.au](http://www.sdi.com.au)  
[www.polawhite.com.au](http://www.polawhite.com.au)

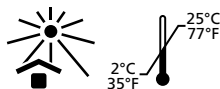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