SDI

Riva Star

INSTRUCTIONS FOR USE

Riva Star is a new generation desensitizer that immediately relieves tooth sensitivity.

INDICATIONS FOR USE:

Desensitising cervical tooth hypersensitivity.

CONTRA-INDICATIONS

- Do not desensitise teeth with carious lesions.
- Do not use on patients undergoing thyroid gland therapy, who have a history of chemical allergies such as silver, fluoride, potassium, iodine, ammonia and methacrylates. Note: If in doubt, consult a doctor or an allergy specialist before use.

PROCEDURE:

- Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
- Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
- Isolate area to be treated with cotton rolls.
- Dry teeth to be treated and adjoining gingival tissues.
- Apply a small amount of Gingival Barrier to the tissues, slightly overlapping the tooth (<1mm) and interproximal spaces.
- Light cure in a **fanning** motion. Note: Do not spot cure Gingival Barrier as this may cause burning of the soft tissues.
- Step 1:** Using the silver brush provided, pierce through the foil of the silver capsule and carefully apply solution to treatment site only.
- Step 2:** Immediately after, use green brush provided, pierce through the foil of the green capsule and apply a generous amount of the solution to treatment site. Apply same solution to any other tooth surfaces that may have come into contact with the solution from the silver capsule. Note: Treatment surface initially appears creamy white, keep applying solution from green capsule until it turns clear.
- Blot dry, remove gingival barrier.
- Discard used capsules and brushes in accordance with local regulations.

DOSAGE:

- Each Step 1 & Step 2 capsules can treat up to 5 teeth per patient.
- Limit of one (1) Step 1 & Step 2 capsule per visit. If needed a second application of Riva Star may be administered after 1 week.

PRECAUTIONS:

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Due to potential staining, it is recommended that anterior teeth not be treated unless diagnosing for caries.**
- Patient must be informed of potential staining prior to treatment.**
- When used as a desensitizer apply Gingival Barrier before applying Step 1 to avoid soft tissue burn.
- Avoid contact of solution from silver capsule (Step 1) with oral tissue, eyes and skin.
- Avoid contact of solution from green capsule (Step 2) with eyes.
- Do not desensitize teeth with carious lesions, unless a glass ionomer is used to cover over the treatment site.
- Keep out of reach of children.
- Do not ingest.
- Solution from silver capsule (Step 1) will stain (see "HANDLING STAINS").
- Wear gloves and eye protection when handling.
- Single use only.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Solution from silver capsule (Step 1) contains 0.06g / mL fluoride ions.
- Not to be used with resin based composite restorative material.
- SDS is available at www.sdi.com.au or contact your SDI representative.

STORAGE AND HANDLING:

- Do not pierce foil of silver capsule or green capsule until ready to use.
- Store in a cool and dark place at temperatures between 2° and 8°C (35° - 45°F).
- Use at room temperature between 20° and 25°C (68° - 77°F).

FIRST AID:

- Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre (Australia, 131 126) or a doctor, or for at least 15 minutes.
- Skin (contact):** Flush with running water (an eschar may form on soft tissue, this should disappear within 24-48 hours)
- Ingestion:** Not expected due to dose volume. If affected, do NOT induce vomiting. Give a glass of milk or water.
- Inhalation:** Not expected due to dose volume. If affected, remove from contaminated area.

HANDLING STAINS:

- Skin:** Apply solution from green capsule with gentle rubbing. Stain will slowly disappear. (If nothing is done stain will gradually disappear after a few days).
- Clothing:** Soak in Napisan* (or other fabric stain remover) overnight and wash per normal.
- Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan* is not the registered trademark of SDI Limited.

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Napisan* is geen geregistreerde handelsmerk van SDI Limited.