

SDI | ULTRAMAT 2

HIGH SPEED TRITURATOR



HIGH SPEED, MULTI-USE TRITURATOR

The Ultramat 2, offers extremely precise, consistent and accurate trituration. Furthermore, it can be used with most encapsulated dental materials. With a high energy output, it has a mixing frequency of 4600 oscillations per minute but uses remarkably little power. Best of all due to its robust and reliable construction, the 5 year warranty assures your purchase is a long lasting investment.

MULTI-USE MIXER

The multi-use Ultramat 2 is suitable for all types and brands of glass ionomer, amalgam and cement capsules.

SAFETY FEATURE

The micro-switch automatically halts trituration when the lid of the Ultramat 2 is lifted.

SIMPLE TOUCH CONTROL SYSTEM

QUIET, LOW VIBRATION

LOW ENERGY USAGE

PRACTICAL AND EASY TO CLEAN



FEATURES

CONSISTENT AND ACCURATE TRITURATION

The Ultramat 2 features a 0-16 second microprocessor-crystal controlled oscillator timer.

HIGH ENERGY OUTPUT

The high energy Ultramat 2 has a mixing frequency of 4600 oscillations per minute +/-1%.

DUAL VOLTAGE

The Ultramat 2 operates on voltages between 120 and 240v, 50 and 60Hz, for compatibility with multiple power voltages and frequencies.

0-16
SECOND
MICROPROCESSOR

4600
OSCILLATIONS

**120 &
240v**

ADVANTAGES

Consistent accurate trituration

5 year limited warranty

0 - 16 second adjustable timer

Quiet, low vibration

Accurate and consistent trituration

For mixing most encapsulated dental materials

Safe and simple to use


TECHNICAL SPECIFICATIONS

Width	217mm	8.5"
Depth	173mm	6.75"
Height	180mm	7.12"
Voltage	120 / 240 volts	50 / 60 Hz
Weight	2.765kg	6.095lbs



5 YEARS
WARRANTY.

5 YEAR WARRANTY

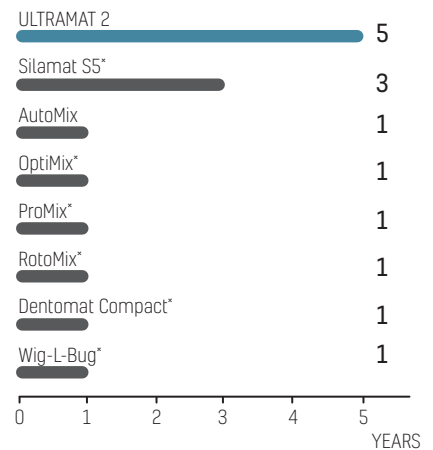


SUITABLE FOR ALL CAPSULES

The Ultramat 2 can be used for the trituration of encapsulated dental restorative materials from SDI Limited as well as from other companies. The following chart lists some examples of SDI encapsulated dental restorative materials.

DENTAL DEVICE (CAPSULES)	TRITUARATION TIMES (SECONDS) USING ULTRAMAT 2 - SPEED SETTING: FIXED
Lojic+	6 ± 1
1, 2, 3 spill Permite / 1, 2, 3 spill GS-80 / Ultracaps+ / Ultracaps S	8 + 1
5 spill Permite / 5 spill GS-80	6 + 1
Riva Light Cure / Light Cure HV / Riva Bond LC / Riva Protect / SeT / Riva Luting / Luting Plus / Riva Self Cure (Regular and Fast)	10



ORDER DETAILS

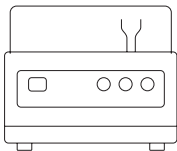

EQUIPMENT

Ultramat 2 (USA - includes power cord for USA)	5546045
Ultramat 2 (All countries - includes power cord for the region the product is shipped to)	5546052
Ultramat X (All countries - includes power cord for the region the product is shipped to)	5546070

*AutoMix, Dentomat Compact, OptiMix, ProMix, RotoMix, Silamat S5, Wig-L-Bug, are not registered trademarks of SDI Limited.



YOUR SMILE. OUR VISION.



C US
ACN 008 075 581



MADE IN AUSTRALIA
by SDI Limited
Bayswater, Victoria 3153
Australia
www.sdi.com.au

AUSTRALIA 1800 337 003
AUSTRIA 00800 0225 5734
BRAZIL 0800 770 1735
FRANCE 00800 0225 5734
GERMANY 0800 100 5759

ITALY 00800 0225 5734
NEW ZEALAND 0800 808 855
SPAIN 00800 0225 5734
UNITED KINGDOM 00800 0225 5734
USA & CANADA 1 800 228 5166