



AURA BULK FILL

UNIVERSAL BULK FILL COMPOSITE




FILL,
PACK
AND GO

THE SIMPLICITY OF BULK FILLING

SPEEDY AND SIMPLE POSTERIOR COMPOSITE RESTORATIONS

- Natural-looking finish with just one universal shade.
- High depth of cure without the need for a capping layer.
- Easy placement in a matter of seconds.
- The waxy consistency is packable and non-sticky to optimise handling.

FILL, PACK AND GO

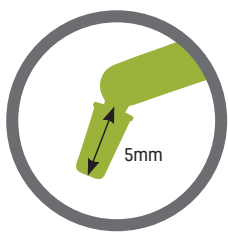


one bulk fill
placement up
to 5mm

+



20sec
LED LIGHT CURE

5mm
BULK FILL

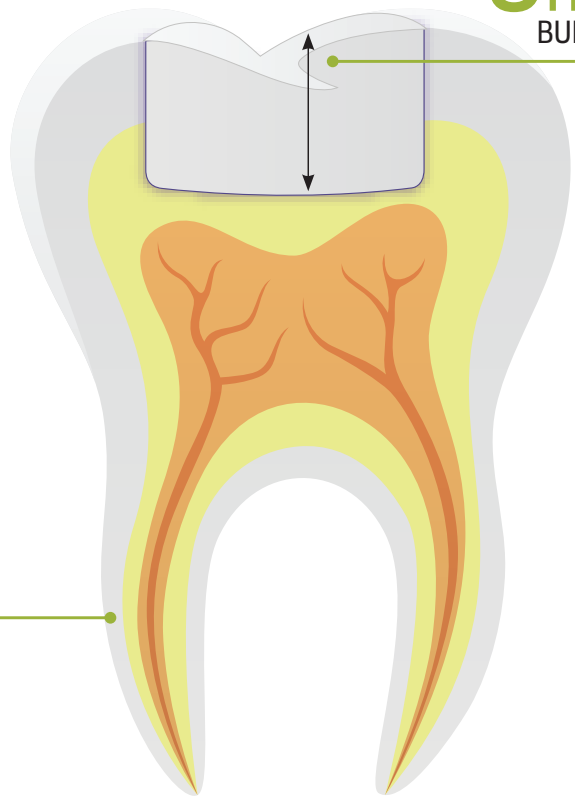

ACHIEVABLE AESTHETICS

CASE STUDIES

DR BOB LOWE [USA]



DR. GREGOR THOMAS [GERMANY]



ONE SHADE. ONE FILL.

LOW VOLUMETRIC SHRINKAGE*

Aura Bulk Fill delivers reduced stress to provide long-lasting restorations.

EXCELLENT FLEXURAL STRENGTH*

Aura Bulk Fill resists major deformations without fracturing.

HIGH COMPRESSIVE STRENGTH*

Aura Bulk Fill resists the high forces of compression in the mouth.

2.21%

115MPa

375MPa

*SDI internal data

INDICATIONS

Posterior restorations

Core buildup

Splinting


Sandwich technique with glass ionomer





DEPTH OF CURE

The opacity of Aura Bulk Fill is a function of the difference in the refractive index of its two components, the filler and the resin.

The curing process alters the refractive index of the resin marginally, to match the refractive index of the filler. This lowers the opacity temporarily, allowing deeper light penetration for a high depth of cure. Upon curing, the indices move apart again to arrive at the ideal opacity for Aura Bulk Fill's brilliant chameleon-like finish.

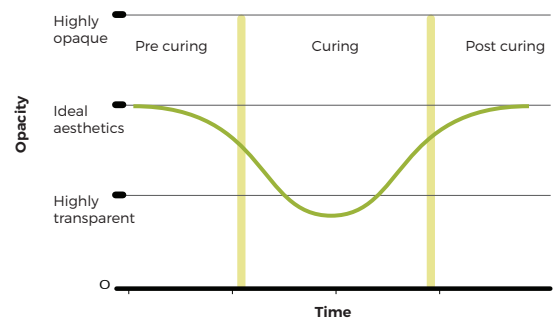

ORDER DETAILS

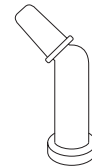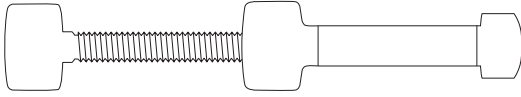

SYRINGES

Syringe Refill - 4g

8560012

COMPLETS

Compleat Refill - 20x 0.25g = 5g total

8565012

» I have been using Aura Bulk Fill for three years now. The resin comes out smooth with no pull back and cures extremely strong. The polish is amazing and my posterior composites have never been better. The characteristics are ideal for any restoration, primary or permanent teeth. I have recommended this product to many associates and I plan to continue using the Aura Bulk Fill as my go to composite. «

DR LIEL GRINBAUM DMD

AURA PRODUCT RANGE

Aura
Aura Easy
Aura Bulk Fill



YOUR SMILE. OUR VISION.

CE
0297

04-2022

M100258 D



+DO36M1002581/

MADE IN AUSTRALIA
by SDI Limited
Bayswater, Victoria 3153
Australia
www.sdi.com.au

AUSTRALIA 1800 337 003
AUSTRIA 00800 0225 5734
BRAZIL 0800 770 1735
FRANCE 00800 0225 5734
GERMANY 0800 100 5759

ITALY 00800 0225 5734
NEW ZEALAND 0800 808 855
SPAIN 00800 0225 5734
UNITED KINGDOM 00800 0225 5734
USA & CANADA 1 800 228 5166