

SDI | ULTRAMAT S

HIGH SPEED TRITURATOR



SO SIMPLE, JUST 3 SIMPLE BUTTONS

There are three time setting options, 6, 8 and 10 seconds to mix all your encapsulated materials. The unit can be stopped at any time by pressing these buttons during operation or just by lifting the lid.

EASY TO CLEAN

The hygienic one surface cover facilitates infection control cleaning and the non-porous plastic surface can be cleaned with any dental disinfectant.

LOW VIBRATION AND EXTREMELY QUIET

Unlike some triturators, the Ultramat S has a low noise output. Minimal vibration of all external parts ensures it will not move from its standing position.

SUITABLE FOR ALL YOUR CAPSULES

The bendable forks are compatible with a broad range of round end capsules (amalgam capsules), and also those with protruding nozzles (glass ionomer cement capsules).



FEATURES

ACCURATE AND CONSISTENT TRITURATION

The Ultramat S features a microprocessor controlled oscillator timer for accurate and consistent trituration.

HIGH ENERGY OUTPUT

Featuring a mixing frequency of 4550 oscillations per minute, the Ultramat S ensures complete mixing of any dental material, while consuming little power.

DUAL VOLTAGE

The Ultramat 2 operates on voltages between 120-240v and 50-60Hz, ensuring compatibility with multiple power voltages and frequencies.

0-10
SECOND
MICROPROCESSOR

4550
OSCILLATIONS

**120 &
240v**

ADVANTAGES

Accurate and consistent trituration

Low vibration and extremely quiet

Simple to use - just 3 buttons

Suitable for all your capsules

Safe and simple to use

Easy to clean

Low power indicator and shut off function

2 years warranty

Dual voltage

TECHNICAL SPECIFICATIONS

Width	217mm	8.5"
Depth	173mm	6.75"
Height	180mm	7.12"
Voltage	120 / 240 volts	50 / 60 Hz
Weight	2.43kg	5.4lbs

LOW POWER INDICATOR AND SHUT OFF FUNCTION

Should the power supply fall outside the required ranges of 100-120V and 220-240V, the power indicator will flash blue and the unit will not function until correct power is supplied to it. This prevents under-mixed materials.

SAFETY FEATURE

The Ultramat S will immediately stop mixing when the transparent lid is opened.




SUITABLE FOR ALL CAPSULES

The Ultramat S can be used for the trituration of encapsulated dental restorative materials from SDI Limited as well as from other companies. The following chart lists some examples of SDI encapsulated dental restorative materials.

DENTAL DEVICE (CAPSULES)	TRITURATION TIMES (SECONDS) USING ULTRAMAT S - SPEED SETTING: FIXED
Lojic+	6 ± 1
1, 2, 3 spill Permite / 1, 2, 3 spill GS-80 / Ultracaps+ / Ultracaps S	8 + 1
5 spill Permite / 5 spill GS-80	8 + 1
Riva Light Cure / Light Cure HV / Riva Self Cure (Regular and Fast) Riva Bond LC / Riva Protect / SeT / Riva Luting / Luting Plus /	10



ORDER DETAILS

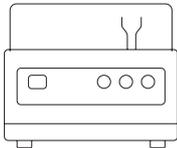

EQUIPMENT

Ultramat S 5546058
(All countries - includes power cord for the region the product is shipped to)

Ultramat 2
(All countries - includes power cord for the region the product is shipped to)

*AutoMix, Dentomat Compact, OptiMix, ProMix, RotoMix, Silamat S5, Wig-L-Bug, are not registered trademarks of SDI Limited.



YOUR SMILE. OUR VISION.



ACN 008 075 581



+D036M55462013

MADE IN AUSTRALIA
by SDI Limited
3-15 Brunsdon Street,
Bayswater, Victoria 3153 Australia
www.sdi.com.au

AUSTRALIA 1800 337 003
AUSTRIA 00800 0225 5734
BRAZIL 0800 770 1735
FRANCE 00800 0225 5734
GERMANY 0800 100 5759

ITALY 00800 0225 5734
NEW ZEALAND +64 21 400 323
SPAIN 00800 0225 5734
UNITED KINGDOM 00800 0225 5734
USA & CANADA 1 800 228 5166