


HIGH STRENGTH, HIGH SILVER SPHERICAL AMALGAM



HIGH STRENGTH SMOOTH HANDLING EXCELLENT POLISHABILITY

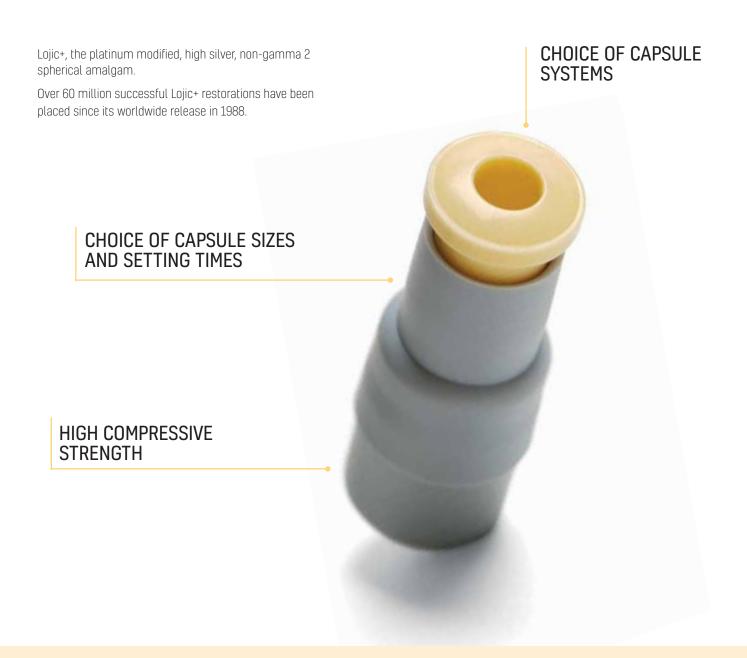

OPTIMAL HANDLING CHARACTERISTICS

Lojic+ mixes homogeneously, is easy to load into the carrier, condenses well, burnishes and carves nicely ⁽³⁾. Dental Advisor stated "no marginal ridge fractures were reported upon matrix removal ⁽³⁾.


Lojic+ is easier to establish proximal contacts than with other single-composition spherical alloys [4].

90% of The Dental Advisor consultants felt the amalgamated mass had the right consistency and that the capsule to capsule consistency was excellent [3].

TOUGH AND DURABLE LONG LASTING RESTORATIONS

HIGH COMPRESSIVE STRENGTH

Lojic+'s high compressive strength minimises the risk of fracturing the restoration. High compressive strength ensures a tough and durable long lasting restoration.

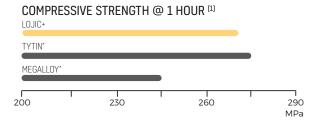


PLATINUM MODIFIED

The addition of platinum improves the compressive and tensile strengths.

LOW MICROLEAKAGE

The microleakage for Lojic+ is considerably lower than other spherical alloys.

Extensive microleakage has been related to increased post operative sensitivity ⁽²⁾.

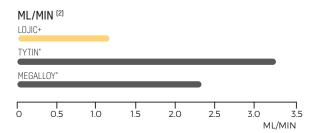

GREATER CHOICE OF SETTING TIMES AND LARGER CHOICE OF CAPSULE SIZES

The working and setting times allow large restorations to be condensed and carved without difficulty.




PHYSICAL PROPERTIES

Composition:	Ag 60.1%, Sn 28.05%, Cu 11.8%, Pt 0.05%, Hg 42.2%
Alloy Particle:	Spherical
Compressive strength @ 1 hour:	270MPa (39,150psi)
Compressive strength @ 24 hours:	520MPa (75,400psi)
Diametral tensile strength @ 1 hour:	35MPa (5,075psi)
Diametral tensile strength @ 24 hours:	50MPa (7,250psi
Static creep @ 7 days:	0.7%
Dimensional change @ 24 hours:	-1µm/cm

CAPSULE SYSTEM

The SDI capsule system is designed to give consistent and accurate mixes every mix.

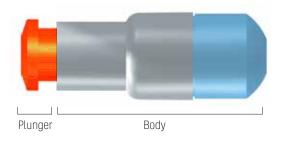

CONVENTIONAL CAPSULES

The plunger denotes the spill size:



The capsule body denotes the setting speed:



ORDER DETAILS



LOJIC+ CAPSULES

Boxes of 50 capsules - One Spill 400mg alloy		
Regular Set	4201303	
Boxes of 50 capsules - Two Spill 600mg alloy		
Regular Set	4202303	
Fast Set	4202202	
Boxes of 50 capsules - Three Spill 800mg alloy		
Regular Set	4202303	
Jars of 500 capsules - One Spill 400mg alloy		
Regular Set	4221303	
Jars of 500 capsules - Two Spill 600mg alloy		
Regular Set	4222303	
Fast Set	4222202	



F	DU	IΡ	MI	ΞN	т

Ultramat 2 [USA - includes power cord for USA]	5546045
Ultramat 2	5546052
(All countries - includes power cord for the region the	
nroduct is shinned to)	

MIXING TIMES

Ultramat 2	
Frequency	Fixed (4550 rpm)
1, 2 & 3 Spill	8 ± 1 sec
5 Spill	6 ± 1 sec

- [1] Source-Published and SDI test data.
- [2] Mahler, D.B., Bryant, R. W., Microleakage of amalgam alloys: An update, JADA September 1996, Vol. 127: 1351-1356
- [3] The Dental Advisor Plus, September/October 1994, Vol. 4, No. 5.
- [4] Lt. Col. Roberts, 56-07 Lojic+ Amalgam (Project 98-03), Brooks Airforce Base, United States Airforce Dental Investigation Services.





MADE IN AUSTRALIA by SDI Limited 3-15 Brunsdon Street, Bayswater, Victoria 3153 www.sdi.com.au

AUSTRALIA 1800 337 003 **AUSTRIA** 00800 022 55 734 **BRAZIL** 0800 770 1735 **FRANCE** 00800 022 55 734 **GERMANY** 0800 100 5759 ITALY 00800 0225 5734

NEW ZEALAND 0800 808 855

SPAIN 00800 022 55 734

UNITED KINGDOM 00800 022 55 734

USA & CANADA 1 800 228 5166

^{*} Tytin and Megalloy are not the registered trademarks of SDI Limited.