

SDI | **POINTS**

DISPOSABLE BRUSH APPLICATORS



DISPOSABLE BRUSH APPLICATORS

Points disposable brush applicators are used to quickly transfer small amounts of material into areas that are difficult to reach. Points are resistant to common solvents in dental materials.

SOLVENT RESISTANT FIBERS

The fiber tips are resistant to harsh environments and solvents.

NON-ABSORBENT, NON-LINTING FIBERS

The fibers will not dislodge during the procedure. The solution is suspended in the fibers until applied.

THREE SIZES

The three sizes of Points, superfine, fine and medium, enable one to pick up different drop sizes.



CHOICE OF COLORS

The various colors are ideal for color coding multi-step procedures.

SOFT FLEXIBLE MATERIAL

Points can be bent for maximum accessibility.

ORDER DETAILS

POINTS

Points purple - superfine 1 x 400 Points purple - superfine	8100122
Points dark green - fine 1 x 400 Points dark green - fine	8100116
Points dark blue - medium 1 x 400 Points dark blue - medium	8100118
Points Dispenser	8100131

Points assortment - all sizes

1 x 100 Points purple - superfine 1 x 100 Points dark green - fine 1 x 100 Points light green - fine 1 x 100 Points dark blue - medium	8100123
Points light green - fine 1 x 400 Points light green - fine	8100117
Points light blue - medium 1 x 400 Points light blue - medium	8100119

INDICATIONS

Sealant application

Placing bonding agents

Applying etchants

Placing cavity liners

Styptic and disclosing solutions

Mop up excess resin

Condensing calcium hydroxide
in root canals

INSTRUCTIONS

- 1 Points can be used either straight or bent



- 2 Dip the Point into the solution and place where required

- 3 Dispose of the Point after the application

Suggestion: When applying more than one solution, use a differently colored Point to avoid confusion



YOUR SMILE. OUR VISION.



MADE IN AUSTRALIA
by SDI Limited
3-15 Brunson Street,
Bayswater, Victoria 3153
www.sdi.com.au

AUSTRALIA 1800 337 003
AUSTRIA 00800 0225 5734
BRAZIL 0800 770 1735
FRANCE 00800 0225 5734
GERMANY 0800 100 5759

ITALY 00800 0225 5734
NEW ZEALAND +64 21 400 323
SPAIN 00800 0225 5734
UNITED KINGDOM 00800 0225 5734
USA & CANADA 1 800 228 5166