# RIVA SILVER

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

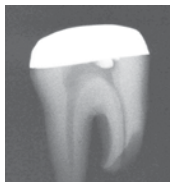
Riva Silver's free movement of fluoride provides benefits to surrounding and adjacent tooth surfaces. Fluoride is held within the glass matrix without being bound by the structure. If the level outside the matrix is lower, then fluoride ions are released.

Conversely, if the fluoride level is higher (eg. from topical fluoride) then fluoride will recharge the glass ionomer matrix. Riva Silver has the ability to be recharged with fluoride from external sources, providing long term protection against recurrent caries. Riva Silver is great for treating high caries-risk patients as part of an overall treatment plan for maximum care.

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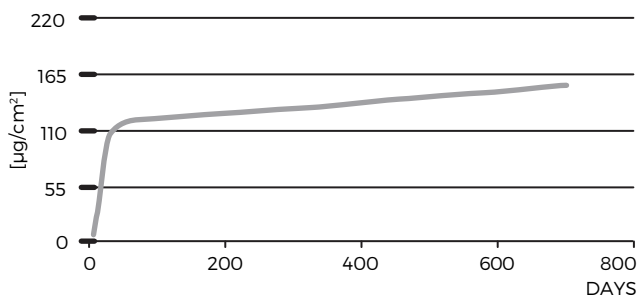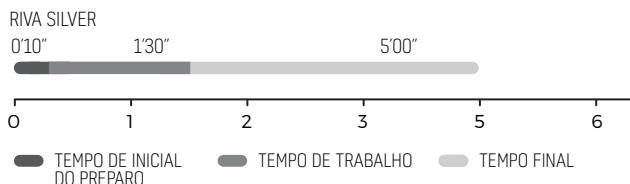## PROCEDIMENTO SIMPLES

Riva Silver's condensable nature allows the operator to quickly and easily place the restoration. Riva Silver can be packed using an amalgam condenser.

## PERFECT FOR NON-COMPLIANT PATIENTS

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
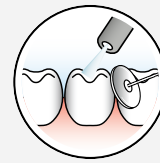
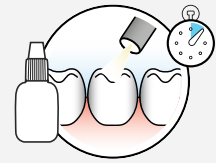
**9** Aplique o Riva Coat and fotopolimerize por 10 segundos

<sup>^</sup> For powder and liquid instructions, please refer to Instructions for Use

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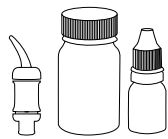\*\* Dados de teste publicados e SDI.

[1] McCabe JF, Al-Naimi OT. Fluoride release of three Riva GI Products Compared with that of a competitor product. University of Newcastle (UK); August 2006.

[2] Al-Naimi, O.T., McCabe, J.F., Comparing the Compressive Strength of Riva Silver GIC with that of other Competitor Products, January 2005, University of Newcastle (UK).

YOUR SMILE. OUR VISION.

MADE IN AUSTRALIA  
by SDI Limited  
Bayswater, Victoria 3153  
Australia 1800 337 003  
[www.sdi.com.au](http://www.sdi.com.au)

**AUSTRIA** 00800 0225 5734  
**BRAZIL** 0800 770 1735  
**FRANCE** 00800 0225 5734  
**GERMANY** 0800 100 5759  
**ITALY** 00800 0225 5734

**NEW ZEALAND** 0800 734 034  
**SPAIN** 00800 0225 5734  
**UNITED KINGDOM** 00800 0225 5734  
**USA & CANADA** 1 800 228 5166

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\* Não são marcas comerciais registradas da SDI Limited.