

For Sculpting and  
Modeling of Composite

## INSTRUCTIONS FOR USE

Dipping Resin from SDI Limited is a HEMA free, BPA free, colourless, low viscosity, unfilled light cured resin. It is designed to eliminate uncured composite sticking to instruments, making it easier to manipulate and sculpt.

**COMPOSITION:**

- Unfilled urethane dimethacrylate resin.

**INDICATIONS:**

- As dipping resin for sculpting and manipulation of composite restorations.

**CONTRAINDICATIONS:**

- Bonding of composite to dentine.
- People with allergies to acrylic resins.

**PROCEDURE:**

1. Dispense 1-2 drops of Dipping Resin into a mixing well and replace cap onto the bottle.
2. Place initial increment of composite to appropriately prepared site.
3. Dip composite instrument into Dipping Resin, sculpt and manipulate the composite to desired shape.
4. Light cure and finish restoration using standard polishing techniques.

**STORAGE AND HANDLING:**

- Dipping resin is light sensitive, do not dispense until ready to use.
- Store at temperatures between 10° and 30°C (50° - 86°F).
- Shelf life: 3 years

**PRECAUTIONS:**

- For professional use only.
- Keep away from sources of direct sunlight.
- Keep out of reach of children. Do not take internally.
- Do not use after expiry date.
- Avoid prolonged contact of uncured Dipping Resin with the skin or oral tissue, as it may cause inflammation of the oral tissues or skin sensitization.
- Avoid contact with eyes.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.

**FIRST AID:**

- **Eye (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol / acetone. Wash thoroughly with water.
- **Ingestion (large amount):** Drink plenty of water. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.

**PACKAGING:**

- Pack of 1 x 5mL bottle



Made in Australia by SDI Limited

Bayswater, Victoria 3153

Australia 1 800 337 003

Austria 00800 0225 5734

Brazil 0800 770 1735

France 00800 0225 5734

Germany 0800 1005759

Ireland 00800 0225 5734

Italy 00800 0225 5734

New Zealand 0800 734 034

Spain 00800 0225 5734

United Kingdom 00800 0225 5734

USA & Canada 1 800 228 5166

[www.sdi.com.au](http://www.sdi.com.au)

[www.polawhite.com.au](http://www.polawhite.com.au)

PAK41306 B

