SDI LUNA Composite restorative material

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Combining nano-hybrid and micro filler particle technology together with non-stick handling, the radiopaque light cure, condensable and low shrinkage composite, Luna offers polishability and strength on an unprecedented scale for anterior and posterior restorations alike.

COMPOSITION:
24% wt. (42% vol.) multifunctional methacrylic esters
76% wt. (58% vol.) inorganic fillers (40nm - 1.5µm)

INDICATIONS:
Direct anterior/posterior restorations and veneers. Core build ups. Indirect inlays, onlays and veneers. Splinting. Composite and porcelain repair. Sandwich technique with glass ionomers.

CONTRAINDICATIONS:

Pulp capping. Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention. Do not use in conjunction with any eugenol-containing materials.

INSTRUCTIONS FOR USE:

- Clean and isolate tooth. Rubber dam is the preferred mode of isolation.
- Prepare a conservative cavity, employing standard techniques.
- Use a calcium hydroxide liner and appropriate glass ionomer cement liner to protect deep cavity preparations in close proximity to the pulp.
- Select composite shade by matching with a clean and moist tooth using the SDI composite shade guide.
- Apply bonding system according to manufacturer's instructions or apply ZipBond Universal adhesive (SDI Limited) or Stae single component dentin/enamel adhesive (SDI Limited).
- Place composite in increments of 2mm or less.
NOTE: If using the syringe system, after each use, turn the screw plunger anti-clockwise half a turn to release any built up pressure.
- Light cure each increment for at least 10 seconds for light shades and at least 20 seconds for dark and/or opaque shades using a high powered LED light such as the Radii Plus (SDI Limited).
- Shape, polish and finish employing standard techniques.

OPTION: Additional polishing using polishing paste: After step 8, polishing paste may be used to give the composite a lustrous finish, using the following steps:

- Apply Polishing Paste (SDI Limited) to a rubber cup or disc.
- Smear a thin layer of paste onto the restoration.
- Add a small amount of water to both the tooth and cup. Polish for 30 seconds at low speed and light pressure.
- Wash and dry.

STORAGE AND HANDLING:

- Store at temperatures between 10° and 25°C (50°-77°F).
- Use at room temperature between 20° and 25°C (68°- 77°F).
- Syringe system: when not in use replace the cap tightly.

PRECAUTIONS:

- Avoid prolonged contact of etchant, adhesive and composite with the skin or oral tissue, as it may cause inflammation of the oral tissues or skin sensitization.
- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Do not expose material to direct light.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID:

- Eye (contact):** Wash thoroughly with water and seek medical attention.
- Skin (contact):** Remove using a cloth or sponge soaked in alcohol. Wash thoroughly with water.
- Ingestion:** Drink plenty of water/milk. Seek medical attention if symptoms persist.
- Inhalation:** No symptoms expected.

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