RIVA APPLICATOR  
RIVA APPLICATOR 2

E  
N  
G  
L  
I  
S  
H

RIVA APPLICATOR  
RIVA APPLICATOR 2

D  
E  
U  
T  
S  
C  
H

RIVA APPLICATOR  
RIVA APPLICATOR 2

D  
U  
T  
C  
H

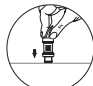
RIVA APPLICATOR  
RIVA APPLICATOR 2

P  
O  
R  
T  
U  
G  
U  
Ê  
S

## INSTRUCTIONS FOR USE

For use with capsulated dental materials.

After pre-cleaning and sterilisation (below), follow these 6 steps:

-  Activate capsule by pressing it down to a hard surface;
-  Place capsule into the high-speed multi-use triturator;
-  Place capsule into the applicator. Make sure the tip of the capsule is facing out;
-  Press applicator two to three times until the material comes out and apply to the patient's tooth;
-  To remove the capsule from the applicator, press the middle button;
-  Refer to sterilsation and cleaning instructions for reprocessing.

## STERILISATION CLEANING & STERILISATION INSTRUCTIONS:

### MANUAL CLEANING:

Step	Solution	Time (minutes)	Temperature	Instruction
1	Water	2	Ambient	Rinse soiled applicator under ambient tap water. Use a soft bristled brush if needed.
2	Hospital Grade Enzymatic detergent	20	Room temperature	Immerse and soak for required time. Clean thoroughly - scrub all external surfaces with a soft bristle brush until all visible soil has been removed. It is important to make sure that all joints and handles are effectively cleaned. Use a small diameter brush or pipe cleaner if needed. Inspect for visible soil on exposed surfaces.
3	Water	2	Warm, as delivered from hot tap	Rinse applicator under warm tap water for required time.
4	Air	As required	Ambient	Allow to air dry in clean area. Blow lumens with clean air using filtered air source or syringe.

### MECHANICAL CLEANING:

The applicator can be mechanically cleaned. However, due to several numbers of brands and quality of automated cleaners on the market, it is not possible to make a claim in compatibility with washer disinfectant meeting all requirements of ISO 15883 series.

The applicator should be positioned to ensure surfaces are exposed to the action of the water spray without dislodgement, and to allow drainage.

For automated cleaning of the applicator, SDI recommends using an approved medical washer disinfectant. Follow instrument washer manufacturer's instructions for use. The applicator should be dry before sterilisation.

### STERILISATION:

A steam steriliser is approved for the applicator. Proceed with sterilisation procedure according to autoclave manufacturer's instructions for use.

Maximum sterilisation temperature: 134°C (273.2 °F).

### Recommended exposure times:

- Pre-Vac pouched cycle type. 30 minutes at 121°C (249.8 °F). 20 minutes dry time.
- Pre-Vac pouched cycle type. 4 minutes at 132°C (269.6 °F). 20 minutes dry time.

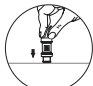
### MAINTENANCE

Regularly apply dental instrument lubricant between applicator handles.

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