polishing paste

E
N
G
L
I
S
H

INSTRUCTIONS FOR USE

The SDI Polishing Paste imparts the appearance of a microfilled to the high strength, anterior / posterior hybrid composite. It contains agglomerated primary crystals of aluminium oxide particles that impart a smooth, luminous finish to all resin surfaces.

COMPOSITION:

- Glycerine
- Aluminium oxide

INDICATIONS:

Suitable for composite, compomer and amalgam polishing. Porcelain, ceramic and gold finishing.

INSTRUCTIONS:

1. Apply the SDI polishing paste to a rubber cup or disc.
2. From the rubber cup or disc, smear a thin layer of the paste onto the restoration.
3. Add a small amount of water to both the tooth and cup or disc.
4. Polish for 30 seconds at low speed and light pressure. SDI Polishing Paste may be used interproximally with floss.
5. Wash and dry.

WARNING: Apply polishing paste at normal room temperature (23°C/74°F).

PRECAUTIONS:

- Use only as a final polish.
- Keep out of reach of children.
- Do not take internally.
- Store at temperatures between 10° and 25°C (50°-77°F).
- When not in use replace the cap tightly.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by on the order of a dentist.

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