ENG ADVANCED TOOTH WHITENING SYSTEM
DEU ANSPRUCHSVOLLES ZAHNAUFHELLUNGSSYSTEM
FRA SYSTÈME AVANCÉ DE BLANCHIMENT DES DENTS
ITA SISTEMA SBIANCANTE EVOLUTO
POL ZAAWANSOWANY SYSTEM DO WYBIELANIA ZĘBÓW.
POR SISTEMA DE CLAREAMENTO DENTAL AVANÇADO
SPA SISTEMA AVANZADO DE BLANQUEAMIENTO DENTAL

Made in Australia by SDI Limited
 3-15 Brunston Street, Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
 Austria 00800 0225 5734
 Brazil 0800 770 1735
 France 00800 0225 5734
 Germany 0800 1005759
 Italy 00800 0225 5734
 Spain 00800 0225 5734
 UK 00800 0225 5734
 USA & Canada 1 800 228 5186

Last Revised: 11-2025

ENG INSTRUCTIONS FOR USE

- WARNING:**
- For professional use only.
 - Patient must wear protective eyewear.
 - Operators must wear gloves, mask and protective eyewear.
 - Do not use on pregnant or lactating women.
 - Do not use on children under 14 years of age.
 - Do not use the Gingival Barrier on any persons having known resin allergies.
 - Do not use Pola Office on any persons having known peroxides allergies.
 - Anyone with a history of chemical allergies is advised to undergo allergy checks before treatment.
 - Do NOT anaesthetize patient.
 - Pola Office Liquid bottle may be under pressure. Remove cap carefully wearing gloves, mask and protective eyewear at all times.
 - Do not allow mixed gel or Pola Office Liquid to come in contact with skin, eyes and soft tissues.
 - Pola Office will not lighten any restorative materials.

FIRST AID:

- SKIN/TISSUES (contact):** Immediately wash thoroughly with water. Apply a slurry of sodium bicarbonate and water to the affected area. Seek medical attention if symptom persists.
- EYES (contact):** Open the eye wide and thoroughly wash for 15 minutes with running water. Seek medical attention.
- INGESTION:** Rinse mouth with water. Gargle with salt water and drink plenty of milk. Seek medical attention if symptoms persist.
- INHALATION (powder):** Remove to fresh air. If breathing is difficult, seek medical attention.

STORAGE:

- Store in cool place (2°-25°C / 35°-77°F)
- AFTER USE, REPLACE BOTTLE/JAR CAPS TIGHTLY

TIPS:

- Patient must have good overall health.
- Seal all exposed roots and restorations.
- Inform patient that results are not guaranteed as the removal of stains varies between patients.
- Acid etching before treatment is NOT recommended.
- Large diameter suction tips are not recommended as they can spread the gel to other areas.
- Mild discomfort in teeth during treatment can be a common side effect.
- If patient experiences sensitivity during light curing, move the tip away slightly. If this doesn't work, remove gel and rinse the affected area. Assess other possible causes.
- Delay any aesthetic restorations for 2 to 4 weeks after treatment.
- Advise patient not to smoke, consume highly coloured, hot or cold foods or beverages for at least 2 days.
- Treat any post-operative sensitivity with a desensitising gel or toothpaste.
- Prescribe Pola Day or Pola Night to maintain brightness if desired.
- SDS available at www.sdi.com.au or contact your regional representative.
- A patient consent form can be requested on marketing@sdi.com.au

INDICATIONS FOR USE:

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

NOTE: To insert the supplied powder scoop leveller into the powder jar, simply remove the moisture seal, then place gently by pushing into the jar. Do not remove the secondary moisture seal from the inside of the jar cap.

INSTRUCTIONS FOR VITAL TOOTH WHITENING

- Determine and record pre-operative shade. Please follow this Vita* shade guide arrangement according to degree of brightness: B1*A1+B2*D2=A2*C1+C2=D4*A3*D3+B3*A3.5+B4*C3+A4*C4
- Clean teeth with ONLY a flour based pumice.
- Place cheek retractors and then cover exposed lip surface with petroleum gel.
- Dry teeth and apply Gingival Barrier to both arches, slightly overlapping enamel and interproximal spaces.
- Light cure in a **fanning motion for 10-20 seconds** until Gingival Barrier is cured.
- Mixing of whitening gel:** Using the supplied empty clear pot (with black lid) as a mixing well, dispense the required amount of powder and liquid. Recommendations are listed in the below table. Please note that these are only recommendations, and that more or less liquid can be added to obtain a desired gel consistency. Replace jar/bottle caps tightly.

1 tooth, 2 applications
 1 level scoop of powder
 5±1 drops of liquid

Single arch, 1 application
 2 level scoops of powder
 10±2 drops of liquid

Dual arch, 1 application
 4 level scoops of powder
 20±3 drops of liquid

- Immediately mix using a brush applicator until gel is homogeneous.
- Apply a **thick** layer of gel to all teeth undergoing treatment.
- Leave gel on for 8 minutes** (Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations)
- Suction off using a surgical aspirator tip.
- Complete Steps 6-10 three times (optional 4 times). Note: gel must be used within 20 minutes of mixing.
- After the last application, suction all the gel off, then wash and apply suction.
- Remove Gingival Barrier by lifting it from one end.

INSTRUCTIONS FOR NON-VITAL TOOTH WHITENING.

- Verify adequate endodontic obturation.
- Isolate tooth with rubber dam.
- Open lingual access to pulp chamber and remove all composite base material and gutta percha 2-3mm apical to the cemento - enamel junction.
- Place glass ionomer base 1mm thick to ensure a barrier between the 'sealed' root canal and whitening gel.
- Place and light cure a thick layer of Gingival Barrier at the gingiva region of the isolated tooth.

- Mixing of whitening gel:** Using the supplied empty clear pot (with black lid) as a mixing well, dispense the required amount of powder and liquid. For two applications on one tooth, it is recommended to dispense 1 scoop of powder and 5 drops of liquid. Please note that this is only a recommendation, and that more or less liquid can be added to obtain a desired gel consistency. Immediately mix using a brush applicator until gel is homogeneous. Replace jar/bottle caps tightly.
- Place the gel into the opened pulp chamber and onto the labial surface.
- Leave gel on for 8 minutes.** (Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations).
- Suction off using a surgical aspirator tip.
- Complete steps 7-9 until the desired shade is reached. Note: gel must be used within 20 minutes of mixing.
- Wash and apply suction after the last application.
- Remove Gingival Barrier and re-seal tooth with appropriate restorative. Note: Recall patients periodically for radiographic and clinical examination.

COMPOSITION

- Pola Office Liquid:** Hydrogen peroxide, Water
- Pola Office Powder:** Thickeners, Catalysts, Dye, Desensitizing agents
- Gingival Barrier:** Methacrylic ester, Silica, Pigment, Butylated hydroxy toluene

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