SDI

# POLA

FOR ALIGNERS

ADVANCED TOOTH  
WHITENING SYSTEM


**WELCOME  
TO YOUR NEW  
POLA SMILE**

PROFESSIONALLY  
USED AND  
RECOMMENDED

---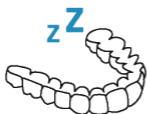

SAFE FOR  
OVERNIGHT WEAR

---

EASY TO USE

---

FLUORIDE  
RELEASING

# INDEX

<b>ENG</b> - ENGLISH .....	4-5
<b>BUL</b> - BULGARIAN .....	6-7
<b>CRO</b> - CROATIAN .....	8-9
<b>DAN</b> - DANISH .....	10-11
<b>DEU</b> - DEUTSCH .....	12-13
<b>NLD</b> - DUTCH .....	14-15
<b>SPA</b> - ESPANOL .....	16-17
<b>EST</b> - ESTONIAN .....	18-19
<b>FIN</b> - FINNISH .....	20-21
<b>FRA</b> - FRANÇAIS .....	22-23
<b>ELL</b> - GREEK .....	24-25
<b>ITA</b> - ITALIANO .....	26-27
<b>LAV</b> - LATVIAN .....	28-29
<b>LIT</b> - LITHUANIAN .....	30-31
<b>NOR</b> - NORWEGIAN .....	32-33
<b>POL</b> - POLSKI .....	34-35
<b>POR</b> - PORTUGUESE .....	36-37
<b>ROM</b> - ROMANIAN .....	38-39
<b>SLO</b> - SLOVAKIAN .....	40-41
<b>SWE</b> - SWEDISH .....	42-43
<b>TUR</b> - TURK .....	44-45

# INSTRUCTIONS FOR USE FOR PATIENTS AND DENTAL PROFESSIONALS

## **POLA NIGHT 10% CARBAMIDE PEROXIDE ADVANCED TOOTH WHITENING SYSTEM**

Pola Night is a high viscosity, neutral pH take home tooth whitening gel. It is a pleasantly flavoured, crystal clear sustained release gel with a unique blend of soothers, conditioners and a water content high enough to further reduce sensitivity.

### **INDICATIONS FOR USE**

- Whitening of discolored vital teeth.
- Whitening of discolored non-vital teeth.

### **CONTAINS HYDROGEN PEROXIDE**

10% carbamide peroxide = 3.6% hydrogen peroxide


### **TREATMENT GUIDELINE**

**Pola Night 10%:** 1 x 2 hours / day to overnight

**Note:** Patients with sensitivity will require a day or two break between treatments. Shortening treatment time is also suggested.

### **PLACEMENT PROCEDURE:**

1. Place a small drop of gel into every compartment of the aligner for the teeth undergoing treatment.
2. Seat the aligner, with the gel around the teeth.
3. Wipe away excess gel in mouth with a tissue or dry soft brush.
4. After treatment, remove aligner. Rinse aligner and mouth with lukewarm water.
5. Brush teeth.

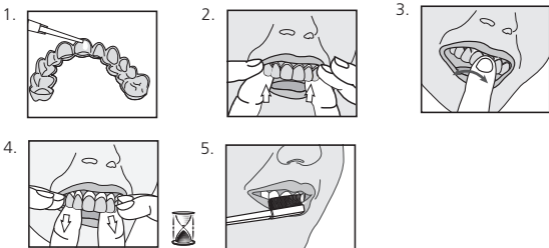## FOR EFFECTIVE RESULTS

- Due to discomfort or sensitivity that can occur as a result of your orthodontic treatment, avoid using this product for the first two weeks after receiving your orthodontic aligners and for the first two days after each new aligner set, as whitening may increase tooth sensitivity.
- Do not eat, drink or smoke during treatment.
- Do not smoke immediately after treatment - wait for at least two hours.
- Foods and drinks containing strong colours should be avoided for at least 48 hours after the treatment period or consumed in moderation.
- Use gel at room temperature.

## STORAGE AND SHELF LIFE

- **Pola Night** - Store in a cool and out of direct sunlight at temperatures between [2°-25°C / 35°-77°F]
- Do not use after expiration date.

## PRECAUTIONS

- To be used under the supervision of a dentist / orthodontist.
- Not to be used on a person under 18 years of age, pregnant or lactating women.
- To be only sold to dental practitioners.
- For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Keep out of reach of children and pets.
- Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using these products.
- Discontinue use if any unusual sensitivity or reactions occur and consult your dentist.
- Do not inject Pola Night into the body.
- CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.

## FIRST AID

**Eye (contact):** Hold eyelids apart and flush with running water for at least 15 minutes. Seek medical advice if symptoms persist.

**Skin (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.

**Ingestion (large amount):** If swallowed, do NOT induce vomiting. Rinse mouth. Drink plenty of milk or water and seek medical advice.

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SDI Limited
SDI Germany GmbH