SDI

POLA ZING

35% carbamide peroxide
advanced tooth whitening system

INSTRUCTIONS FOR USE
FOR PATIENTS AND
DENTAL PROFESSIONALS

ENGLISH

Pola Zing is a 35% carbamide peroxide, high viscosity, neutral pH take home tooth whitening gel. It is a pleasantly flavoured, crystal clear sustained release gel with a water content high enough to reduce sensitivity.


- INDICATIONS FOR USE:**
- Whitening of discoloured vital teeth.
 - Whitening of discoloured non-vital teeth.

COMPOSITION: 35% Carbamide peroxide
40% Water
0.5% Flavour
24.5% Thickening agents

NET WEIGHT:
Depending on kit purchased, 1.3g syringes or 3g syringes.
Please refer to outside of box.

HYDROGEN PEROXIDE EQUIVALENT:
Pola Zing 35% Carbamide Peroxide = 11.7% Hydrogen Peroxide

- INSTRUCTIONS:**
- Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
 - Seat the tray, with the gel around the teeth.
 - Wipe away excess gel in mouth with a tissue or dry soft brush.
 - After 15 minutes of treatment, remove tray. Rinse tray and mouth with lukewarm water.
Note: Treatment time can be up to 30 minutes if required.
 - Brush teeth.

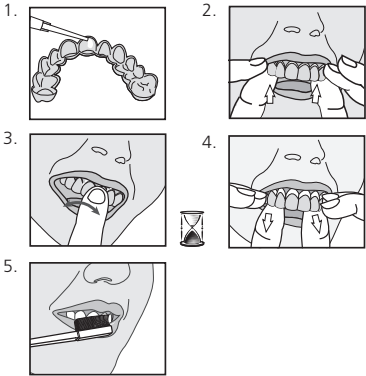- STORAGE AND HANDLING:**
- Store product in the refrigerator (2°-8°C / 35°-45°F).
 - Do not use after expiration date.
 - Use product at room temperature.

- PRECAUTIONS:**
- To be used under the supervision of a dentist.
 - Do not eat, drink or smoke during treatment.
 - Do not smoke immediately after treatment - wait for at least two hours.
 - Food and drinks containing strong colours should be avoided or consumed in moderation during the entire treatment period.
 - Keep out of reach of children and pets.
 - Not to be used by pregnant or lactating women.**
 - Certain countries have age restrictions where tooth whitening products are not to be used:** Canada: under 12 years of age; New Zealand: under 16 years of age.
 - If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult an oral health professional.
 - Consult your oral health professional before prolonged use of this product.
 - Avoid swallowing the cosmetic or part thereof.
 - Avoid contact of the product with the eye.
 - Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.
 - Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using this product.
 - Discontinue use if any unusual sensitivity or reactions occur and consult your dentist.
 - Do not inject Pola Zing into the body.
 - CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST.**
 - SDS available at www.sdi.com.au or contact your regional representative.

- FIRST AID:**
- Eye (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
 - Skin/Tissue (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
 - Ingestion (large amount):** Drink plenty of milk or water and seek medical advice.

Made in Australia by SDI Limited
3-15 Brunson Street
Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au
www.polawhite.com.au

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