SDI POLISHING PASTE

INSTRUCTIONS FOR USE

E
N
G
L
I
S
H

The SDI Polishing Paste imparts the appearance of a microfilled to the high strength, anterior / posterior hybrid composite. It contains agglomerated primary crystals of aluminium oxide particles that impart a smooth, luminous finish to all resin surfaces.

INDICATIONS:

Suitable for composite, compomer and amalgam polishing. Porcelain, ceramic and gold finishing.

INSTRUCTIONS:

1. Apply the SDI Polishing Paste to a rubber cup or disc.
2. From the rubber cup or disc, smear a thin layer of the paste onto the restoration.
3. Add a small amount of water to both the tooth and cup or disc.
4. Polish for 30 seconds at low speed and light pressure. SDI Polishing Paste may be used interproximally with floss.
5. Wash and dry.

WARNING: Apply Polishing Paste at normal room temperature (23°C/74°F).

PRECAUTIONS:

- Use only as a final polish.
- Keep out of reach of children.
- Do not take internally.
- Store at temperatures between 10° and 25°C (50°-77°F).
- When not in use replace the cap tightly.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

Polishing Paste

Reorder 8100150
4g syringe

Made in Australia by SDI Limited
3-15 Brunson Street
Bayswater, Victoria 3153

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au

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