SDI RIVA STAR

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PAK41310 E

Made in Australia by SDI Limited
 3-15 Brunson Street, Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
 Austria 0800 0225 5734
 Brazil 0800 770 1735
 France 0800 0225 5734
 Germany 0800 1005759
 Italy 0800 0225 5734
 Spain 0800 0225 5734
 UK 0800 0225 5734
 USA & Canada 1 800 228 5166

Last Revised: 08-2025

ENG INSTRUCTIONS FOR USE

Riva Star is a tooth desensitising agent, anti-carries agent and caries detector.

INDICATIONS FOR USE:

- Desensitising cervical tooth hypersensitivity.
- Arresting Caries.
- Detecting Caries.

CONTRA-INDICATIONS

Do not use on patients undergoing thyroid gland therapy, who have a history of chemical allergies such as silver, fluoride, potassium, iodine, ammonia and methacrylates
Note: If in doubt, consult a doctor or an allergy specialist before use.

PROCEDURE: A. DESENSITISING CERVICAL TOOTH HYPERSENSITIVITY:

- Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
- Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
- Isolate area to be treated using cotton rolls.
- Dry teeth to be treated and adjoining gingival tissues.
- Apply a small amount of Gingival Barrier (SDI Limited) to gingival tissue (supplied in kit), slightly overlapping the tooth (<1mm) and interproximal spaces. Light cure in a fanning motion. Note: Do not spot cure Gingival Barrier as this may cause burning of the soft tissues. Alternatively, a rubber dam can be used for isolation.
- Protect any exposed gingival tissues and lips with petroleum jelly/cocoa butter taking care not to contaminate treatment site.
- Riva Star Step 1:** Using the **silver** brush provided, pierce through the foil of the **silver** capsule and carefully apply solution to treatment site only.
- Riva Star Step 2:** Immediately after, use **green** brush provided, pierce through the foil of the **green** capsule and apply a generous amount of the solution to treatment site until the **creamy white precipitate turns clear** (if this precipitate does not turn clear staining may result). **Note:** Apply same solution from **green** capsule to any other tooth surfaces that may have come into contact with the solution from the Riva Star Step 1 capsule (**silver** capsule).
- Blot dry.
- Remove all protective/isolation materials used in the mouth.
- Discard used capsules and brushes in accordance with local regulations.

B a) ARRESTING CARIES WITHOUT RESTORING TOOTH:

Note: If using on carious lesions or demineralised dentition, treatment site will stain or slightly discolour. Patients must be informed of this prior to treatment. Ideally keep treatment away from aesthetic zone.

- Isolate area to be treated using cotton rolls.
- Dry teeth to be treated and adjoining gingival tissues.
- Apply a small amount of Gingival Barrier (SDI Limited) to gingival tissue (supplied in kit), slightly overlapping the tooth (<1mm) and interproximal spaces. Light cure in a fanning motion. Note: Do not spot cure Gingival Barrier as this may cause burning of the soft tissues. Alternatively, a rubber dam can be used.
- Protect any exposed gingival tissues and lips with petroleum jelly/cocoa butter taking care not to contaminate treatment site.
- Remove any food debris using an excavator.
- Riva Star Step 1:** Using the **silver** brush provided, pierce through the foil of the **silver** capsule and carefully apply solution to treatment site only.
- Riva Star Step 2:** Immediately after, use **green** brush provided, pierce through the foil of the **green** capsule and apply a generous amount of the solution to treatment site until the **creamy white precipitate turns clear** (if this precipitate does not turn clear staining may result). In some cases, a residual yellowish matt surface may remain. **Note:** Apply same solution from **green** capsule to any other tooth surfaces that may have come into contact with the solution from the Riva Star Step 1 capsule (**silver** capsule).
- Remove all protective/isolation materials used in the mouth.
- Reschedule patient to return for permanent restoration procedure. For deciduous teeth, wait for teeth to exfoliate.
- Discard used capsules and brushes in accordance with local regulations.

B b) ARRESTING CARIES AND RESTORING TOOTH:

Note: Avoid aesthetic zone until you have become familiar with the following technique.

- Isolate area to be treated using cotton rolls.
- Dry teeth to be treated and adjoining gingival tissues.

- Apply a small amount of Gingival Barrier (SDI Limited) to gingival tissue (supplied in kit), slightly overlapping the tooth (<1mm) and interproximal spaces. Light cure in a fanning motion. **Note:** Do not spot cure Gingival Barrier as this may cause burning of the soft tissues. Alternatively, a rubber dam can be used.

- Protect any exposed gingival tissues and lips with petroleum jelly/cocoa butter taking care not to contaminate treatment site.

- Prepare a conservative cavity without removing affected dentine.

- Create a moat (trough) around the dentine enamel junction with a #3 round bur into sound dentine.

- Apply 37% phosphoric acid, e.g. Super Etch (SDI Limited), to site for five (5) seconds - rinse with water for 10 seconds.

- Riva Star Step 1:** Using the **silver** brush provided, pierce through the foil of the **silver** capsule and carefully apply solution to treatment site only.

- Riva Star Step 2:** Immediately after, use **green** brush provided, pierce through the foil of the **green** capsule and apply a generous amount of the solution to treatment site until the **creamy white precipitate turns clear** (if this precipitate does not turn clear staining may result). In some cases, a residual yellowish matt surface may remain. **Note:** Apply same solution from Riva Star Step 2 **green** capsule to any other tooth surfaces that may have come into contact with the solution from the Riva Star Step 1 capsule (**silver** capsule).

- Wash thoroughly with water for at least 10 seconds and air dry. **Note:** Do not desiccate surface if a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited) is to be used.

- Discard used capsules and brushes in accordance with local regulations.

- a) Place a restorative material over the treatment site, such as a self-cure glass ionomer cement, Riva Self Cure (SDI Limited).**

- Note:** Polyacrylic conditioning is not required before placing a conventional glass ionomer.

- OR**

- b) If a composite material, such as Aura Bulk Fill (SDI Limited) is to be used to restore the tooth, first a conventional glass ionomer cement should be placed as a liner, followed by a glass ionomer based bonding agent such as Riva Bond LC (SDI Limited) or resin-based adhesive, such as Zipbond Universal (SDI Limited).**

- Note:** If a resin-based composite is directly placed over the Riva Star site, staining may occur due to any free silver ions becoming light activated, hence darkening.

- Remove all protective/isolation materials used in the mouth.

- C. DETECTING CARIES:**

- Isolate area to be treated using cotton rolls.

- Dry teeth to be treated and adjoining gingival tissues.

- Apply a small amount of Gingival Barrier (SDI Limited) to gingival tissue (supplied in kit), slightly overlapping the tooth (<1mm) and interproximal spaces. Light cure in a fanning motion. **Note:** Do not spot cure Gingival Barrier as this may cause burning of the soft tissues. Alternatively, a rubber dam can be used.

- Protect any exposed gingival tissues and lips with petroleum jelly/cocoa butter taking care not to contaminate treatment site.

- Riva Star Step 1:** Using the **silver** brush provided, pierce through the foil of the **silver** capsule and carefully apply solution to site to be diagnosed and leave for 1 minute.

- Riva Star Step 2:** Immediately after, use **green** brush provided, pierce through the foil of the **green** capsule and apply a generous amount of the solution to treatment site until the **creamy white precipitate turns clear** (if this precipitate does not turn clear staining may result). In some cases, a residual yellowish matt surface may remain. **Note:** Apply same solution from **green** capsule to any other tooth surfaces that may have come into contact with the solution from the Riva Star Step 1 capsule (**silver** capsule).

- Wash thoroughly with water for at least 10 seconds and air dry. **Note:** Do not desiccate surface if a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited) is to be used.


- Remove all protective/isolation materials used in the mouth.

- Discard used capsules and brushes in accordance with local regulations.

- Note:** Treated sites should begin to turn brown within 24 hours where there are early stages of carious lesions. A dental curing light can be used to accelerate the visible colour change. Stained sites can be reduced by using a judicious polishing

procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement.

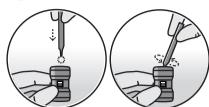Fig.1 How to pierce capsule:

- Use colour coded brush to pierce foil on capsule.
- In circular motion, push foil to the edge of the opening.

DOSAGE:

- Each Riva Star Step 1 and Step 2 capsule can treat up to 5 teeth per patient.
- Limit of one (1) Riva Star Step 1 & one (1) Riva Star Step 2 capsule per visit. If needed a second application of Riva Star may be administered after 1 week.

PRECAUTIONS:

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Due to potential staining, it is recommended that anterior teeth not be treated unless diagnosing for caries.
- Patient must be informed of potential staining prior to treatment
- Apply Gingival Barrier (SDI Limited) before applying Riva Star Step 1 to avoid soft tissue burn
- Avoid contact of Riva Star Step 1 solution (**silver** capsule) with oral soft tissue, eyes and skin. If accidental contact occurs, refer to First Aid below.
- Avoid contact of Riva Star Step 2 solution (**green** capsule) with eyes.
- When used as a desensitiser, do not desensitise teeth with soft dentine as staining may occur **Note:** If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited).
- Keep out of reach of children.
- Do not ingest.
- Riva Star Step 1 Solution (**silver** capsule) will stain (see "Handling Stains").
- Wear gloves and eye protection when handling.
- Single use only.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Riva Star Step 1 Solution (**silver** capsule) contains 0.06g / mL fluoride ions.
- SDS is available at www.sdi.com.au or contact your SDI representative.

- STORAGE AND HANDLING:**
- Do not pierce foil of Riva Star **silver** capsule or **green** capsule until ready to use.
- Store in a cool and dark place at temperatures between 2°C and 8°C.
- Use at room temperature between 20°C and 25°C.
- FIRST AID:**
- Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes. Seek for medical attention.
- Skin/oral soft tissue (contact):** Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
- Ingestion:** Not expected due to dose volume. If affected, do NOT induce vomiting. Give a glass of milk or water. For advice, contact a Poison Information Centre or a doctor.
- Inhalation:** Not expected due to dose volume. If affected, remove from contaminated area.

STORAGE AND HANDLING:

- Do not pierce foil of Riva Star **silver** capsule or **green** capsule until ready to use.
- Store in a cool and dark place at temperatures between 2°C and 8°C.
- Use at room temperature between 20°C and 25°C.

FIRST AID:

- Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes. Seek for medical attention.
- Skin/oral soft tissue (contact):** Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
- Ingestion:** Not expected due to dose volume. If affected, do NOT induce vomiting. Give a glass of milk or water. For advice, contact a Poison Information Centre or a doctor.
- Inhalation:** Not expected due to dose volume. If affected, remove from contaminated area.

HANDLING STAINS:

- Skin:** Apply Riva Star Step 2 solution (**green** capsule) with gentle rubbing. Stain will slowly disappear. Otherwise, if nothing is done, the stain should gradually disappear after a few days.
- Clothing:** Soak contaminated clothing in a diluted fabric stain remover and wash per normal.
- Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

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