INSTRUCTIONS FOR USE (BOTTLE)

Riva Star Aqua is an ammonia free, aqueous next generation desensitizer that immediately relieves tooth sensitivity.

INDICATIONS FOR USE:

- Treatment of dental hypersensitivity.
- For use in adults over the age of 21.

CONTRA-INDICATIONS

- Do not use in patients with ulcerative gingivitis, stomatitis or patients undergoing thyroid gland therapy.
- Do not use on patients with known sensitivity to silver or other heavy - metal ions, fluoride, potassium or iodine. **Note:** If in doubt, consult a doctor or an allergy specialist before use. Patients with more than five affected teeth, patients having had full mouth gingivectomies and patients showing abnormal skin sensitization in daily circumstance are recommended for exclusion.

PROCEDURE:

1. Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
2. Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
3. Isolate area to be treated using cotton rolls.
4. Protect lips with petroleum jelly/cocoa butter taking care not to contaminate treatment site.
5. Riva Star Aqua Step 1: Dispense one (1) drop of solution (**blue** label) onto non-absorbent mixing pad or dampen dish.
6. Carefully apply solution to treatment site only, a medium sized micro brush, such as Points (SDI Limited) is recommended.
7. Riva Star Step 2: Immediately after, dispense two (2) drops of solution (**green** label) onto fresh non-absorbent mixing pad.
8. Apply a generous amount of the solution to treatment site, a medium sized micro brush, such as Points (SDI Limited) is recommended. Apply same solution to any other tooth surfaces that may have come into contact with the solution from Riva Star Aqua Step 1 (**blue** label). **Note:** Treatment surface initially appears creamy white, keep applying Riva Star Step 2 (**green** label) solution until it turns clear.
9. Blot dry. **IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.
10. Clean dappen dishes and discard used mixing pads and micro-brushes in accordance with local regulations.

DOSAGE:

- One (1) drop of Riva Star Aqua Step 1 and two (2) drops of Riva Star Step 2 can treat up to 5 teeth per patient.
- Limit of one (1) drop of Riva Star Aqua Step 1 and two (2) drops of Riva Star Step 2 drops per visit. If needed a second application of Riva Star Aqua may be administered after 1 week.

PRECAUTIONS:

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Patient must be informed of potential staining prior to treatment.
- Due to potential staining, it is recommended that anterior teeth not be treated.
- Patient must be informed of potential staining prior to treatment.
- Avoid contact of solution from Riva Star Aqua Step 1 (**blue** label) with oral soft tissue, eyes, skin and clothing. If accidental contact occurs refer to the "FIRST AID" below.
- Avoid contact of solution from Riva Star Step 2 (**green** label) with eyes.
- Do not desensitize teeth with soft dentine as staining may occur. **Note:** If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement (such as Riva Self-Cure, SDI Limited).
- Keep out of reach of children.
- Do not ingest.
- Solution from Riva Star Aqua Step 1 (**blue** label) will stain (refer to "HANDLING STAINS" below).
- Wear gloves and eye protection when handling.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Solution from Riva Star Aqua Step 1 (**blue** label) contains 0.06g / mL fluoride ions.
- SDS is available at www.sdi.com.au or contact your SDI representative.

STORAGE AND HANDLING:

- Store bottles upright in the packaging provided.
- Store between 20°and 25°C (68°- 77°F) and use at room temperature.
- Do not refrigerate products.
- Do not dispense solution from blue label or green label bottles until ready to use.
- Replace cap on bottle and seal tightly immediately after use. **IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

FIRST AID:

- **Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes. Seek medical attention if symptoms persist.
- **Skin/oral soft tissue (contact):** Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
- **Ingestion:** Do NOT induce vomiting. Give a glass of milk or water. Seek immediate medical attention.
- **Inhalation:** If affected, remove from contaminated area.

HANDLING STAINS:

- **Skin:** In case of skin contact with Riva Star Aqua Step 1 solution (**blue** label), apply Riva Star Step 2 solution (**green** label) with gentle rubbing. Stain will slowly disappear. Otherwise, if Riva Star Step 2 solution is not applied, the stain should gradually disappear after a few days.
- **Clothing:** Remove contaminated clothing and soak in Napisan* (or other fabric stain remover) overnight and wash per normal.
- **Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan* is not the registered trademark of SDI Limited.

PACKAGING:

- Each kit contains:
- 1 x Riva Star Aqua Step 1 (**blue** label) bottle 1.5ml
- 1 x Riva Star Step 2 (**green** label) bottle 3.0ml

US Patents granted.

Made in Australia by SDI Limited
3-15 Brunston Street
Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au
www.rivastar.com

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Napisan* no es una marca registrada de SDI Limited.