INSTRUCTIONS FOR USE  
(BOTTLE)

ENGLISH

Riva Star is a dental cariostatic and desensitizing agent that detects and arrests carious lesions.

## INDICATIONS FOR USE:

- A. Desensitising cervical tooth hypersensitivity.
- B. Arresting Caries
- C. Detection of carious lesions

## CONTRA-INDICATIONS

Do not use on patients undergoing thyroid gland therapy, who have a history of chemical allergies such as silver, fluoride, potassium, iodine and ammonia. Note: If in doubt, consult a doctor or an allergy specialist before use.

Note: If the product is used in carious lesions or demineralized areas, the treatment site may present a darkening effect. The patients should be informed before treatment.

## PROCEDURE:

## A. DESENSITISING CERVICAL TOOTH HYPERSENSITIVITY:

1. Assess the condition of the teeth. Only treat teeth with sound dentin and enamel unless a restorative material is will be used to cover the treated area.
2. Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and adjacent teeth.
3. Dry the area to be treated and isolate it using rubber dam or a light curable gingival barrier (SDI Limited).
4. Protect any exposed gingival tissues and lips with petroleum jelly/ cocco butter taking care not to contaminate treatment site. When not using rubber dam, isolate adjacent area with cotton rolls.
- NOTE 1:** Make sure that the area where the Riva Star will be applied is well isolated to prevent product flowing to the adjacent teeth.
- NOTE 2:** If the product seeps into a tooth that has decayed tissue, it may stain.
- Riva Star Step 1:** Dispense one (1) drop of the solution on a non-absorbent mixing pad or Dappen dish. (**grey label**)
5. Carefully apply solution to treatment site only, a medium sized micro brush, such as Points (SDI Limited) is recommended.
6. Apply a generous amount of the solution to the treatment site. It is recommended to use a medium micro brush such as Points (SDI). Apply the same solution to other surfaces dental products that may have come into contact with the **Riva Star Step 1** solution (**grey label**).
- Note:** The surface being treated will initially be creamy white, continue applying the Riva Star Step 2 solution (**green label**) until it is clear.
7. Blot dry.
10. Remove all protective/isolation materials used in the mouth.
11. Clean dappen dishes and discard used mixing pads and micro-brushes in accordance with local regulations.

**IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

## B a). ARRESTING CARIES WITHOUT RESTORING TOOTH:

**Note:** If used on carious lesions or demineralised dentition, treatment site will stain or slightly discolour. Patients must be informed of this prior to treatment. Ideally keep treatment away from aesthetic zone.

## 1. Using a non-fluoridated prophylaxis paste in a rubber, clean the teeth to be treated and the adjacent teeth.

2. Dry area to be treated and isolate it using rubber dam or a light curable gingival barrier (SDI Limited).
3. Protect any exposed gingival tissues and lips with petroleum jelly/ cocco butter taking care not to contaminate treatment site. When not using rubber dam, isolate area with cotton rolls.
4. Remove any food debris using an excavator.
- NOTE 1:** Make sure that the region where the Riva Star will be applied is well insulated to prevent product flow to the adjacent teeth.
- NOTE 2:** If the product seeps into a tooth that has decayed tissue, it may stain.
- Riva Star Step 1:** Dispense one (1) drop of the solution on a non-absorbent mixing pad or dappen dish. (**grey label**)
- 6. Carefully apply the solution only to the treatment site, keeping in contact with the carious lesion for 60 seconds, it is recommended to use a medium micro brush such as Points (SDI).**
- 7. Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (**green label**) onto fresh non-absorbent mixing pad or dappen dish.
8. Apply a generous amount of the solution to treatment site, a medium sized micro brush recommended, such as Points (SDI Limited). Apply same solution to any other tooth surfaces that may have come into contact with the **Riva Star Step 1** solution (**grey label**). Note: Treatment surface initially appears creamy white, keep applying **Riva Star Step 2** solution (**green label**) until it turns clear. (It is possible that a residual yellowish matt or brownish surface may remain).
9. Remove all protective/isolation materials used in the mouth.
10. Blot dry.

**IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

11. Reschedule patient to return for permanent restoration procedure. For deciduous teeth, wait for teeth to exfoliate
12. Clean dappen dishes and discard used mix pads and micro-brushes in accordance with local regulations.

## B b). ARRESTING CARIOS INJURIES AND RESTORING THE TOOTH

**Note:** If the product is used in carious lesions or demineralized areas, at the treatment site it may have a darkening effect. The patients should be informed before treatment. Preferably, keep the treatment away from the aesthetic area.

## 1. Using a non-fluoridated prophylaxis paste in a rubber, clean the teeth to be treated and the adjacent teeth.

2. Dry area to be treated and isolate it using rubber dam or a light curable gingival barrier (SDI Limited).
3. Protect any exposed gingival tissues and lips with petroleum jelly/ cocco butter taking care not to contaminate treatment site. When not using rubber dam, isolate area with cotton rolls.

**NOTE 1:** make sure that the region where the Riva Star will be applied is well insulated to prevent product flow to the adjacent teeth.

**NOTE 2:** If the product seeps into a tooth that has decayed tissue, it may stain.

4. Prepare a conservative cavity without removing affected dentine.

5. To help prevent staining at the restoration margin, its recommended to create a moat (rough) around the dentine enamel junction with a #3 round bur into sound dentine.

6. Apply 37% phosphoric acid, e.g. Super Etch (SDI Limited), to site for 5 seconds - wash with water then dry.

**Riva Star Step 1:** Dispense one (1) drop of solution (**grey label**) onto non-absorbent mixing pad dappen dish.

8. Carefully apply the solution to the treatment site only. It is recommended to use a medium micro brush such as Points (SDI).

**Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (**green label**) onto fresh non-absorbent mixing pad or dappen dish.

10. Apply a generous amount of the solution to treatment site, a medium sized micro brush is recommended, such as Points (SDI Limited).

Apply same solution to any other tooth surfaces that may have come into contact with the **Riva Star Step 1** solution (**grey label**).

Note: Treatment surface initially appears creamy white, keep applying **Riva Star Step 2** solution (**green label**) until it turns clear. (It is possible that a residual yellowish matt surface may remain).

11. Wash thoroughly with water for at least 10 sec. and air dry.

**Note:** Do not desiccate surface if a self curing glass ionomer is to be used.

12. Clean dappen dishes and discard used mixing pads and micro-brushes in accordance with local regulations.

13. a) Place self-cure glass ionomer cement such as Riva Self Cure or Riva Self Cure HV (SDI Limited).

**Note:** Polycrylic conditioning is not required before placing glass ionomer.

OR

b) If a composite material, such as Aura Bulk Fill (SDI Limited) is to be used to restore the tooth, first a conventional self-cure glass ionomer cement such as Riva Self Cure or Riva Self Cure HV (SDI Limited) should be placed as a liner, followed by a glass ionomer based bonding agent such as Riva Bond LC (SDI Limited) or resin based adhesive, such as Zipbond Universal (SDI Limited). As a final layer, a resin-based composite, such as Aura Bulk Fill (SDI Limited) may be placed to finish off the restoration.

**Note:** If a resin-based composite is directly placed over the Riva Star site, staining may occur due to any free silver ions becoming light-activated, hence darkening.

**IMPORTANT :** Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

14. Remove all protective/isolation materials used in the mouth.

## C. DETECTION OF CARIOUS LESIONS

**Note:** If the product is used in carious lesions or demineralized areas, the treatment site it may present a darkening effect. The patient should be informed before treatment.

1. Dry the area to be treated and isolate it using rubber dam or a light cure light-cured gingival barrier (SDI Limited).

2. Protect any exposed gingival tissues and lips with petroleum jelly/ cocco butter taking care not to contaminate treatment site. When not using rubber dam, isolate adjacent area with cotton rolls.

**NOTE 1:** Make sure that the area where the Riva Star will be applied is well isolated to prevent product flowing to the adjacent teeth.

**NOTE 2:** If the product seeps into a tooth that has decayed tissue, it may stain.

**Riva Star Step 1:** Dispense one (1) drop of the solution (**grey label**) on a non absorbent mixing pad or Dappen dish.

4. Carefully apply solution to site to be diagnosed, a medium sized micro brush is recommended, such as Points (SDI Limited).

**Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (**green label**) onto fresh non-absorbent mixing pad or Dappen dish.

6. Apply a generous amount of the solution to treatment site, a medium sized micro brush is recommended, such as Points (SDI Limited). Apply same solution to any other tooth surfaces that may have come into contact with the **Riva Star Step 1** solution (**grey label**).

7. Wash thoroughly with water for at least 10 seconds and air dry. Note: Do not desiccate surface if a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited), is to be used.

8. Remove all protective/isolation materials used in the mouth.

9. Clean dappen dishes and discard used mix pads and brushes in accordance with local regulations

**IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

10. Clean dappen dishes and discard used mix pads and brushes in accordance with local regulations

**Note:** Treated sites should begin to turn brown within 24 hours where there are early stages of carious lesions. A dental curing light can be used to accelerate the visible colour change. Stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited).

**DOSAGE:**

• One (1) drop of **Riva Star Step 1** and two (2) drops of **Riva Star Step 2** can treat up to 5 teeth per patient.

• Limit of one (1) drop of **Riva Star Step 1** and two (2) drops of **Riva Star Step 2** per visit. If needed a second application of Riva Star may be administered after 1 week.

## PRECAUTIONS:

• For professional use only.

• Do not repeat treatment on pregnant or lactating women.

• Due to potential staining, it is recommended that anterior teeth not be treated unless diagnosing for caries. When diagnosed with caries, please follow procedure B (Arresting Caries) of this Instructions for Use.

• Patient must be informed of potential staining prior to treatment.

• Avoid contact of solution from **Riva Star Step 1 bottle (grey label)** with oral soft tissue, eyes, skin and clothing. If accidental contact occurs, refer to "FIRST AID" below.

• Avoid contact of solution from **Riva Star Step 2 bottle (green label)** with eyes.

• When used as a desensitizer, do not desensitize carious lesions as staining may occur. Note: If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited).

• Keep out of reach of children.

• Do not ingest.

• Solution from **Riva Star Step 1 bottle (grey label)** will stain (see "HANDLING STAINS").

• Wear gloves and eye protection when handling.

• Do not use after expiry date.

• Caution: Federal Law restricts this device to sale by or on the order of a dentist.

• Solution from **Riva Star Step 1 bottle (grey label)** contains 0.06g / mL fluoride ions.

• SDS is available at [www.sdi.com.au](http://www.sdi.com.au) or contact your SDI representative.

## STORAGE AND HANDLING:

• Store bottles upright in the packaging provided.

• Store between 2° and 25°C (35°-77°F).

• Do not use the product straight out of the fridge and use at room temperature between 20° and 25°C (68°-77°F).

• Do not dispense solution from grey label bottle or green label bottle until ready to use.

• Place cap on bottle and seal tightly immediately after use.

**IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

**FIRST AID:**

• **Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes. Seek for medical attention.

• **Skin/oral soft tissue (contact):** Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).

• **Ingestion:** If affected, do NOT induce vomiting. Give a glass of milk or water. For advice contact a Poisons Information Center or a doctor at once. Seek medical attention.

• **Inhalation:** If affected, remove from contaminated area.

## HANDLING STAINS:

• **Skin:** In case of skin contact with **Riva Star Step 1 (grey label)**, apply **Riva Star Step 2** solution (**green label**) with gentle rubbing. Stain will slowly disappear. Otherwise, if Riva Star step 2 is not applied, the stain should gradually disappear after a few days.

• **Clothing:** Remove contaminated clothing and soak in Napisan® (or other fabric stain remover) overnight and wash per normal.

• **Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

## PACKAGING:

Each Kit contains:

1 x **Riva Star Step 1 (grey label)** bottle 1.5 mL

1 x **Riva Star Step 2 (green label)** bottle 3.0 mL

US Patents granted.

Made in Australia by SDI Limited  
3-15 Brunsdon Street  
Bayswater, Victoria 3153  
Australia

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1. Seque la zona a tratar y aíslela con un dique de goma ou una barrera gingival fotopolimérrizable (SDI Limited).