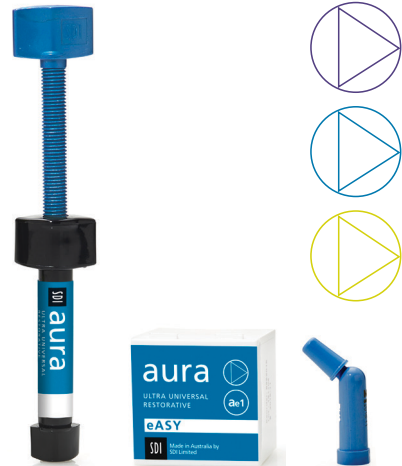
aura eASY

ULTRA UNIVERSAL
RESTORATIVE MATERIAL
INSTRUCTIONS FOR USE

PRM1133_A

aura eASY

ULTRA UNIVERSAL
RESTORATIVE MATERIAL
INSTRUCTIONS FOR USE

E
N
G
L
I
S
H

INTRODUCTION

aura eASY is a light cured, radiopaque, high strength, universal composite designed to meet most of your aesthetic restorative needs with just four simple shades. Each **aura eASY** shade is perfectly equally spaced out in colour intensity so that you can customise shades if needed. The **aura eASY** range has a 3mm depth of cure which reduces the standard number of incremental layering when restoring deep cavities.

INDICATIONS

- Anterior restorations
- Posterior restorations
- Indirect inlays, onlay and veneers
- Core buildup
- Splinting
- Sandwich technique with glass-ionomer

CONTRA-INDICATIONS

- Pulp capping.
- Do not use in conjunction with any eugenol containing materials.
- Anyone with allergy to acrylics

SHADE SELECTION

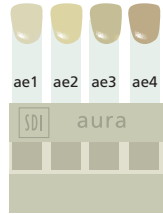
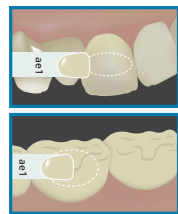
1. Do not over dry the tooth before shade matching.
2. Use the **aura eASY** shade guide to select your shade before placement of rubber dam.
3. When the adjacent teeth are significantly different in shade, try to match the lighter shade.
4. For visual confirmation of shade selected, place small amounts of composite on a surrounding tooth surface and light cure before bonding

Approximate reference to
VITA classical
A1-D4® shades

A1/B1 ≈ ae1
A2/B2 ≈ ae2
A3/B3 ≈ ae3
A3.5/B4 ≈ ae4

Shade selection for non layering technique:

The **aura eASY** range is most suitable for non-layering requirements where high aesthetics are not necessary. To choose an **aura eASY** shade, look at the overall body of the tooth using the **aura eASY** shade tabs.

PLACEMENT PROCEDURE:

1. Isolate tooth.
2. Prepare cavity using standard technique for composite resin restorations.
- Note:** In cases where pulp protection is necessary, use a calcium hydroxide liner.
3. Apply a dentin/enamel bonding agent such as SDI Go! as per manufacturer's instructions.
4. Place selected shade(s) of **aura eASY** composite in increments of 3mm or less.
5. Light cure each increment for at least 20 seconds (460 - 480nm wavelength, high power LED curing light).
6. Finish restoration using standard technique.

STORAGE AND HANDLING

- Store at temperatures between 10° and 25°C (50°-77°F).
- Use at room temperature between 20° and 25°C (68°- 77°F).
- Syringe system: when not in use replace the cap tightly.

PRECAUTIONS

- Avoid prolonged contact of **aura eASY** with the skin or oral tissue, as it may cause inflammation of the oral tissues or skin sensitization.
- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Do not expose material to direct light.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID

- **Eye (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol. Wash thoroughly with water.
- **Ingestion:** Rinse mouth thoroughly. Drink plenty of water/milk. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.

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