# SDI POLA RAPID

6% HYDROGEN PEROXIDE
ADVANCED TOOTH WHITENING SYSTEM

## ENGLISH

### INSTRUCTIONS FOR USE

Pola Rapid is a hydrogen peroxide based in-office tooth whitening system requiring minimal chair time. Contains desensitizers and fluoride to maximize patient comfort. The dual barrel syringe system mixes as the gel is applied directly to the tooth.

#### INDICATIONS:

- Whitening of discoloured vital teeth.
- Whitening of discoloured non-vital teeth.

#### NET WEIGHT:

Depending on kit purchased. Please refer to outside of box:
Pola Rapid syringe: 3.2g / 2.8mL gel

#### CONTRA-INDICATIONS:

- Do not use on pregnant or lactating women.
- Do not use on children under 18 years of age.
- Do not use Pola Rapid on any persons having known peroxides allergies.
- Pola Rapid will not lighten any restorative materials.
- Do not use on patients with extremely sensitive teeth.

#### PRECAUTIONS:

- Contains sodium fluoride and hydrogen peroxide.
- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured.
- Afterwards to be provided to the consumer to complete the cycle of use.

- Irritant. Avoid contact with eyes and skin.
- For professional use only.
- Do not remove syringe from wrapper until ready to use - Pola Rapid is heat and light sensitive.
- Keep out of reach of children and pets.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.
- Pola Rapid syringes may be under pressure - remove cap and attach nozzle with caution.
- Patient must wear protective eyewear.
- Operators must wear gloves, mask and protective eyewear.
- Anyone with a history of chemical allergies is advised to undergo allergy checks before treatment.
- Do NOT anaesthetize patient.

#### FIRST AID:

- For advice, contact a Poisons Information Centre or a doctor (if onco). Urgent hospital treatment is likely to be needed.
- SKIN/TISSUES/HAIR (contact)**: If skin or hair contact occurs remove contaminated clothing and flush skin and hair with running water. Apply a slurry of sodium bicarbonate and water to the affected area seek medical attention if symptom persists.
- EYES (contact)**: If in eyes, hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes.
- INGESTION**: If swallowed, do NOT induce vomiting. Immediately give a glass of water. Rinse mouth with water. Gargle with salt water and drink lots of milk. Seek medical attention if symptoms persist.
- INHALATION** : No symptoms expected.

**STORAGE AND HANDLING:**
Store in a cool place (2°-25°C / 35°-77°F). For best results, keep refrigerated. Use product at room temperature. Do not use after expiration date.

#### TIPS:

- Select appropriate candidate. Note: Not everyone is suitable for in-office whitening. E.g. patient with translucent teeth or in the Vita® C Group.
- Patient must have good overall health.
- Seal all exposed roots and restorations.
- Inform patient that results are not guaranteed as the removal of stains varies between patients.
- Acid etching before whitening is NOT recommended.
- Large diameter suction tips are not recommended as they can spread the gel to other areas.
- Mild discomfort in teeth during treatment can be a common side effect.
- If patient experiences sensitivity during light curing, move the tip slightly away from the tooth. If this doesn't work, remove gel and rinse the affected area. Assess other possible causes.
- Delay any aesthetic restorations for 2 to 4 weeks after treatment.
- Advise patient not to smoke, consume highly coloured, hot or cold foods or beverages for at least 2 days.
- Treat any rest-operative sensitivity with a desensitising gel or toothpaste.
- Prescribe Pola Day or Pola Night to maintain brightness if desired.
- If patient consent form is available for download at [www.sdi.com.au](http://www.sdi.com.au)

#### INSTRUCTIONS FOR VITAL TOOTH WHITENING:

- Determine and register a cor antes de iniciar o tratamento. Usar a escala de cor Vita® por sua vasta seleção de cores: B1•A1•B2•D2•A2•C1•C2•D4•A3•D3•B3•A3.5•B4•C3•A4•C4
- Clean teeth with ONLY a flour based pumice.
- Place cheek retractors and then cover exposed lip surface with petroleum gel.
- Apply attach a mixing nozzle to the Pola Rapid syringe away from patient, and insert the plunger. Dispense a small amount of gel onto a mixing pad until a uniform blue gel is extruded.
- Using the nozzle as a guide, directly apply a thin layer of gel to all teeth undergoing treatment. A thin layer will help prevent the gel from running.
- Leave gel on for 15 minutes (Optional: A curing light may be used at this stage. Radi Xpert or Radi Plus (SDI Limited).
- Remove gel and suction off. Note: It is normal for a residue layer of gel to remain on the surface of the teeth.
- Complete steps 5- 7 three times (optional 4 times).
- After the last application, suction all the gel off, then wash and apply suction.
- Record the final shade.

- Isolate tooth effectively.
- Verify adequate endodontic obturation.
- Use protective barrier to prevent peroxide from reaching the periodontal ligament.
- Avoid acid etching as periodontal ligament irritation may result.
- Advise the patient that a whitened non-vital tooth will not look the same as a vital tooth.
- Recall patients periodically to examine the whitened teeth (clinically & radiographically).

#### INSTRUCTIONS FOR NON-VITAL TOOTH WHITENING:

- Determine and record pre-operative shade. Please follow this Vita® shade guide arrangement according to degree of brightness: B1•A1•B2•D2•A2•C1•C2•D4•A3•D3•B3•A3.5•B4•C3•A4•C4
- Open lingual access to pulp chamber and remove all composite, base material and gutta percha 2-3 mm apical to the CE junction. Place a glass ionomer/zinc phosphate base, at least 1 mm thick to ensure a barrier between the "sealed" root canal and whitening gel.
- Place the Pola Rapid gel into the opened pulp chamber and onto the labial surface.
- Leave gel on for 15 min (Optional: A curing light may be used at this stage. Radi Xpert or Radi Plus (SDI Limited).
- NOTE: If using a curing light and treating inner and outer surfaces, 2 curing lights can be used (one on each side of tooth) to activate Pola Rapid.
- Suction off the gel.
- Repeat steps 3- 5 until desired shade is achieved.
- After the last application, suction and wash the gel off.
- Record the final shade.

\*Vita shade guide is a trademark of VITA Zahnfabrik, H. Rauter GmbH & Co. KG, D-79713 Säckingen, Germany

<p>Made in Australia by SDI Limited</p> <p>3-15 Brunson Street,</p> <p>Bayswater, Victoria 31053, Australia</p>	<p><b>EC DEFP</b></p> <p>SDI Germany GmbH</p> <p>Hansestrasse 85</p> <p>51149 Cologne</p> <p>Germany</p> <p>+49 2203 9255 0</p>
-----------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------

Australia 1 800 337 033
Austria 0800 2255 5734
Brazil 0800 710 1735
France 0800 025 5734
Germany 0800 1005759
Italy 0800 025 5734
Spain 0800 025 5734
United Kingdom 0800 025 5734
USA & Canada 1 800 228 5166
www.sdi.com.au
www.polawhite.com.au

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\*Vita Nuanceguide er et varemærke for VITA Zahnfabrik, H. Rauter GmbH & Co. KG, D-79713 Säckingen, Tyskland