SDI RIVA STAR AQUA

INSTRUCTIONS FOR USE (CAPSULES)

ENGLISH

Riva Star Aqua is an ammonia free new generation desensitiser that immediately relieves tooth sensitivity.

INDICATIONS FOR USE:

Desensitising cervical tooth hypersensitivity.

CONTRA-INDICATIONS

Do not use in patients undergoing thyroid gland therapy, who have a history of chemical allergies such as silver, fluoride, potassium and iodine. Note: If in doubt, consult a doctor or an allergy specialist before use.

PROCEDURE:

- Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
- Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
- Isolate area to be treated using cotton rolls.
- Protect any exposed gingival tissues and lips with petroleum jelly/ cocoa butter taking care not to contaminate treatment site.
- Riva Star Aqua Step 1:** Using the **silver blue** brush provided, pierce through the foil of the **silver blue** capsule and carefully apply solution to treatment site only.
- Riva Star Aqua Step 2:** Immediately after, use **green** brush provided, pierce through the foil of the **green** capsule and apply a generous amount of the solution to treatment site **until the creamy white precipitate turns clear** (if this precipitate does not turn clear you may reapply the solution).
- Riva Star Aqua Step 3:** Immediately after, use **green** brush provided, pierce through the foil of the **green** capsule and apply a generous amount of the solution to treatment site **until the creamy white precipitate turns clear** (if this precipitate does not turn clear you may reapply the solution).
- Blot dry.
- Apply a protective/isolation materials used in the mouth.
- Discard used capsules and brushes in accordance with local regulations.

DOSAGE:

- Each Riva Star Aqua Step 1 and Step 2 capsule can treat up to 5 teeth per patient.
- Limit of one (1) Riva Star Aqua Step 1 and one (1) Step 2 capsule per visit. If needed a second application of Riva Star Aqua may be administered after 1 week.

Fig.1 How to pierce capsule

- Use colour coded brush to pierce foil on capsule.
- In a circular motion, push foil to the edge of the opening.

PRECAUTIONS:

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Patient must be informed of potential allergic treatment.
- Avoid contact of Riva Star Aqua Step 1 solution (**silver blue** capsule) with oral soft tissue, eyes and skin. If accidental contact occurs, refer to "FIRST AID" below.
- Avoid contact of Riva Star Step 2 solution (**green** capsule) with eyes.
- Do not desensitize teeth with soft dentine as staining may occur.
- Note:** If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement (such as Riva Self-Cure, SDI Limited).
- Keep caps until ready to use.
- Do not ingest.
- Riva Star Aqua Step 1 Solution (**silver blue** capsule) will stain (see "HANDLING STAINS").
- Wear gloves and eye protection when handling.
- Single use only.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Riva Star Aqua Step 1 Solution (**silver blue** capsule) contains 0.06g / mL fluoride ion ready to use.
- SDS is available at www.sdi.com.au or contact your SDI representative.

STORAGE AND HANDLING:

- Do not pierce foil of Riva Star Aqua **silver blue** capsule or **green** capsule until ready to use.
- Store in a cool and dark place at temperatures between 2° and 25°C (35° - 77°F).
- Use at room temperature between 20° and 25°C (68° - 77°F).

FIRST AID:

- Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre (Australia, 131 126) or a doctor, or for at least 15 minutes.
- Skin/oral soft tissue (contact):** Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
- Ingestion:** Not expected due to dose volume. If affected, do NOT induce vomiting. Give a glass of milk or water. For advice contact a Poisons Information Centre or a doctor.
- Inhalation:** Not expected due to dose volume. If affected, remove from contaminated area.

HANDLING STAINS:

- Skin:** In case of skin contact with Riva Star Aqua Step 1 (**silver blue** capsule), apply Riva Star Step 2 (**green** capsule) with gentle rubbing. Stain will slowly disappear. Otherwise, if Riva Star Step 2 is not applied, the stain should disappear gradually after a few days.
- Clothing:** Remove contaminated clothing and soak in Napisan* (or other fabric stain remover) overnight and wash per normal.
- Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan* is not the registered trademark of SDI Limited.

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