SDI RIVA CONDITIONER

E
N
G
L
I
S
H

POLYACRYLIC ACID CONDITIONER INSTRUCTIONS FOR USE

Riva Conditioner is a mild solution of polyacrylic acid developed to partially remove the smear layer of dentin and enamel. This optimizes adhesion of the glass ionomer cement to tooth structure.

INDICATIONS:

Glass ionomer cement restorations requiring a conditioning material prior to application.

CONTRAINDICATIONS:

Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

INSTRUCTIONS FOR USE:

1. After tooth preparation, apply Riva Conditioner to the prepared surfaces.
2. Leave for 10 seconds.
3. Rinse thoroughly with water.
4. Remove excess water. Keep moist. Avoid contamination. IMPORTANT: Be careful not to completely dry the surface.
5. Apply glass ionomer according to manufacturer's instructions.
6. Replace cap immediately after use.

STORAGE AND HANDLING

- Do not dispense Riva Conditioner until ready to use.
- Store in a dark place at temperatures between 10° and 25°C (50° - 77°F).
- Use at room temperature between 20° and 25°C (68° - 77°F).

PRECAUTIONS:

- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID

- Eye (contact): Wash thoroughly with water and seek medical attention.
- Skin (contact): Wash thoroughly with water.
- Ingestion: Drink plenty of water/milk. Seek medical attention if symptoms persist.
- Inhalation: No symptoms expected.

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