# SDI shade modification kit

E  
N  
G  
L  
I  
S  
H

## INSTRUCTIONS FOR USE

### DESCRIPTION OF MATERIAL:

SDI's Tints and Masking Agents simulate a life like, natural tooth. The natural tooth color is derived from the color saturated dentin in conjunction with light penetrating the translucent enamel and undergoing reflection, transmission and absorption. The SDI Shade Modification system simply and quickly achieves optimum esthetics with paint on, single component, light cured liquid Tints and Masking Agents. The system contains nine translucent Tints and four Masking Agents. Tints: Blue, Yellow, Clear, Red, White, Orange, Grey, Brown, Green. Masking agents: Grey, Universal, White, Dark Yellow.

### COMPOSITION:

65% wt methacrylic acid esters  
35% wt pyrogenic silica


### INDICATIONS FOR USE:

Suitable for enamel, dentin, composite, compomer, porcelain, etched glass ionomer and metal.

### INSTRUCTIONS:

Clean and isolate tooth

1. Etch tooth surface with Super Etch 37% phosphoric acid, for 20 seconds.
2. Wash thoroughly with water.
3. Remove excess water. Keep moist.
4. Apply Stae dentin/enamel adhesive to saturate all internal surfaces, or other bonding agent according to manufacturer's instructions.
5. Apply appropriate SDI Tint or Masking Agent (< 0.5 mm). The following diagram can be used as a guide to creating a life like restoration.

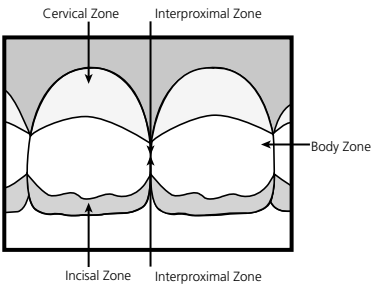1. Cervical Zone - Red, Yellow, Orange or Brown
2. Body Zone - Grey, Yellow, Universal
3. Interproximal Zone - Differing between patients
4. Incisal Zone - Grey, Blue. Darker colors in gingival areas become lighter towards the incisal edge

6. Light cure for 20 seconds. Additional Tints or Masking Agent layers may be applied.
7. Cover with a thin layer of translucent composite such as Glacier Incisal, Wave, or other restorative material according to manufacturer's instructions.

### SPECIAL NOTES:

1. The masking shade should match both the dentin and the restorative material shade.
  2. Masking Agents should be placed as deep as possible.
  3. Too much masking results in a flat lifeless appearance.
  4. A thin layer of the restorative material should be placed over the Masking Agent followed by a final layer of incisal/enamel shade.
  5. Tints should be used under the final layer of the restorative material.
  6. Do not mix Tints and Masking Agents with the restorative material of choice as this will create porosity.
  7. Selecting shade by consensus increases the chance of success as everyone has different color perception.
  8. Lipstick should be removed.
  9. Tooth to be matched should be clean and moist.
  10. Use natural light as well as room light.
  11. Blue color bib is recommended.
  12. Take brief looks. Do not stare.
  13. Select shade in natural, neutral surroundings.
  14. Color perception is affected by the color of walls and patient's clothing.
  15. Tints can be mixed with the clear Tints or with each other to obtain degrees of saturation and different tones.
- IMPORTANT: Apply Tints and Masking Agents at normal room temperature (23°C/74°F).

### CONDITIONS OF STORAGE:

- Do not refrigerate.
- Store at temperatures between 10° and 25°C (50°-77°F).
- Do not use after expiry date.
- Do not expose material to direct light.

### PRECAUTIONS:

- Avoid prolonged contact with the skin or oral tissue, as it may cause inflammation of the oral tissues or skin sensitization.
- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Do not expose material to direct light.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.

### FIRST AID:

- Eye (contact): Wash thoroughly with water and seek medical attention.
- Skin (contact): Remove using a cloth or sponge soaked in alcohol. Wash thoroughly with water.
- Ingestion: Drink plenty of water/milk. Seek medical attention if symptoms persist.
- Inhalation: No symptoms expected.

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