RIVA APPLICATOR

RIVA APPLICATOR, RIVA APPLICATOR 2, RIVA APPLICATOR PRO

ENG	CAPSULE APPLICATOR
BUL	АПЛИКАТОР ЗА КАПУСИ/И
CES	APLIKÁTOR KAPŠÍ
DAN	KAPSELAPPLICATOR
DEU	KAPSELAPPLICATOR
ELL	ΕΠΙΛΟΓΕ ΕΒΑΘΜΩΣΗ ΚΑΠΟΥΛΑΣ
FIN	KAPSULIAPPLIKATTORI
FRA	APPLICATEUR POUR CAPSULE
HUN	KAPSZLA APPLICATOR
ITA	APPLICATORE DI CAPSULE
KOR	캡슐어플리케이터
LIT	KAPŠULINIS APPLICATORIUS
SLK	KAPŠULIN APPLICATOR
POL	APLIKATOR KAPSULKI
POR	APLICADOR DE CAPSULAS
RON	APLICATOR DE CAPSULE
SLK	APLIKATOR KAPŠUL
SPA	APLICADOR DE CAPSULAS
SWE	KAPSULAPPLIKATOR

Made for SDI Limited 3-15 Burdon Street, Baywater, Victoria 3153, Australia	EC REP SDI Germany GmbH Bismarckstrasse 5 51149 Cologne, Germany T: +49 2203 9255 0 F: +49 2203 9255 5734 UK 0800 0225 5734 USA & Canada 1 800 225 5166
--	--

ENG INSTRUCTIONS FOR USE

INTENDED USE:
For the application of capsulated dental materials by dental professionals.
DESCRIPTION:
The Riva Applicator is used to apply dental filling materials directly from the capsule into the restorative surface.

COMPOSITION: Riva Applicator is 304 stainless steel. Riva Applicator 2 is made of anodized aluminum. Riva Applicator Pro is 316L Stainless Steel coated with fluorinated alkyl silane.

INSTRUCTIONS

1. Ensure that the applicator plunger is retracted by pressing the release button.
2. After insertion of the capsule, remove the applicator from the capsule and insert capsule into capsule lock of the applicator.
3. Make two priming strokes with the handle.
4. Squeeze the lever slowly maintaining enough pressure to ensure the material is forced into the tooth or into the prepared cavity.
5. Press the release button to retract the plunger.
6. Remove the used capsule from the applicator by pulling the capsule out of the capsule lock.
7. Refer to cleaning and sterilisation instructions for reprocessing.

- NOTE:** The applicator may be wiped.
WARNING: Reprocess Riva Applicator between patients to avoid cross contamination.
- CONTRA-INDICATIONS / SIDE EFFECTS:**
- Not to be used on patients or by operators with known history of metal contact allergy.
 - Do not continue to use if signs of allergy arise.
 - Take care not to pinch hands or fingers during operation.

CLEANING & STERILISATION INSTRUCTIONS

Step	Solution	Time (minutes)	Temperature	Instruction
1	Water	2	Ambient	Rinse solid applicator under ambient tap water. Use a soft bristled brush if needed.
2	Hospital Grade Enzymatic detergent	20	Room temperature	Immerse and soak for required time. Clean thoroughly - scrub all external surfaces with a soft bristled brush and all visible soil has been removed. It is important to make sure that all pits and angles are effectively cleaned. Use a small diameter brush or pipe cleaner to remove debris from visible soil on exposed surfaces.
3	Water	2	Warm, as required	Rinse applicator under warm tap water for required time.
4	Air	As required	Ambient	Allow to air dry in clean area. Blow hands with clean air flow from source or air gun.

MECHANICAL CLEANING:
The applicator can be mechanically cleaned. However, due to several numbers of brands and quality of automated cleaners on the market, it is not possible to make a claim in compatibility with washer/detergent meeting all requirements of ISO 15883 series. The applicator is not designed to be cleaned in automated cleaners exposed to the action of the water spray without dislodgement, and to allow automatic cleaning of the applicator. The manufacturer recommends using an approved medical washer/detergent. Follow instrument washer manufacturer's instructions for use.

ULTRASONIC BATH
The applicator may be cleaned with an ultrasonic bath prior to sterilisation. Proceed with manufacturer's instructions for use. May be used to surface scratches and residual clearing discoloration. Remove cleaning discoloration with wet sandpaper.

STERILISATION:
A steam steriliser is approved for the applicator. The applicator should be dry with steam in the steriliser. Do not use the applicator for autoclave sterilisation. Proceed with manufacturer's instructions for use. Maximum sterilisation temperature: 134°C (273.2°F).

Recommended exposure times:

- Pre-Vac: pouch cycle type, 30 minutes at 121°C (249.8°F), 20 minutes dry time.
- Pre-Vac: pouch cycle type, 4 minutes at 132°C (269.6°F), 20 minutes dry time.

MAINTENANCE AND STORAGE
Regularly apply dental instrument lubricant between handles (Riva Applicator 2 only). Store applicator in a clean and dry place.

WARRANTY
The Manufacturer, extends the following warranty periods against defects in materials or workmanship to the original purchaser of this product:

- One year warranty for Riva Applicator 2 and Riva Applicator Pro.
- Two year warranty for Riva Applicator 2 and Riva Applicator Pro.

The Manufacturer agrees to accept any defects which develop within the warranty period, either by repair or replacement, at its option. This warranty is valid providing factory inspection indicates that any such defect developed during normal and proper use subject to the modes below.

WARRANTY CONDITIONS - PLEASE READ CAREFULLY

- Please fill out the warranty card included in kit and send promptly back to SDI with proof of purchase. Failure to do so may void your warranty.
- Alternatively go to www.sdi.com.au/warranty to register your warranty.

CLAIMS FOR WARRANTY

- a) Claims for damage in shipment should be filed promptly with the transportation company.
- b) All shipments claimed defective can only be returned to the Manufacturer with the written consent of the Manufacturer. All returned products must be accompanied by a full description of discrepancy or malfunction.
- c) Only SDI authorised service personnel are allowed to carry out repairs to the Riva Applicator, Riva Applicator 2 and Riva Applicator Pro. Manufacturer shall release from all obligations under the Manufacturer's repair or replacement policy if the repair is made by persons other than Manufacturer's own or authorised service personnel.
- d) Use only SDI parts exclusively to replace defective components. The product warranty does not cover any damage resulting from the use of third-party replacement parts.
- e) **Manufacturer shall be released from all obligations under this warranty in the event of improper installation, damage due to negligence, or obvious misuse including but not limited to damage from liquids and improper autoclaving.**
- f) Manufacturer shall in no event be liable for any consequential damages arising from the use of the product.
- g) Only the above stated warranty shall be applicable, and all other warranties, expressed or implied, are hereby excluded. All correspondence concerning the product should specify the model and serial number.

CAUTION: Federal Law restricts this device to sale by or on the order of a dentist. Where this device is sold or used by a person other than a dentist, please report this incident to the manufacturer of this product and to the competent authority in your state or country.

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