# RIVA COAT

<b>ENG</b>	LIGHT CURED COATING MATERIAL
<b>BUL</b>	СВЕТЛИНО ВТВЪРДЕН ПОКРИВЕН МАТЕРИАЛ ЗА СЪКЛЮИОНОМЕРНИ ПРОДУКТИ
<b>DAN</b>	LET HÆRDET BELÆGNINGSMATERIALE TIL GLASIONOMER PRODUKTER
<b>DEU</b>	LICHTHÄRTENDES SCHUTZMATERIAL FÜR GLASIONOMERPRODUKTE
<b>ELL</b>	ΕΛΑΦΡΥ ΣΚΛΗΡΥΜΕΝΟ ΥΛΙΚΟ ΕΠΙΣΤΡΩΣΗΣ ΓΙΑ ΠΡΟΪΟΝΤΑ ΥΑΛΙΟΝΟΜΕΡΟΥΣ
<b>FIN</b>	KEVYESTI KOVETETTU PINNOLTEMATERIAALI LASH-IONOMEERITUOTTELLE
<b>FRA</b>	VERNIS DE PROTECTION PHOTOPOLYMERISABLE POUR VERRRE IONOMÈRE
<b>ITA</b>	ACCA FOTOPOLIMERIZZABILE PER LA PROTEZIONE DEI MATERIALI VETRO-IONOMERICI
<b>LIT</b>	LACCA FOTOPOLIMERIZZABILE PER LA PROTEZIONE DEI MATERIALI VETRO-IONOMERICI
<b>NLD</b>	LICHTHARDENDE COATING MATERIAAL VOOR GLASIONOMEERPRODUCTEN
<b>POL</b>	SMIATECOUTWARDZALNY LAKIER OCHRONNY DLA CEMENTÓW SZKŁO-IONOMEROVYCH
<b>POR</b>	SELANTE DE SUPERFÍCIE FOTOPOLIMERIZÁVEL PARA IONÓMERO DE VIDRO
<b>SPA</b>	RECUBRIMIENTO FOTOCURABLE PARA PRODUCTOS DE IONÓMERO DE VIDRIO

PAK11135 K

Made in Australia by SDI Limited  
3-15 Brunson Street, Bayswater, Victoria 3153, Australia

Australia 1 800 337 003  
Austria 00800 0225 5734  
Brazil 0800 770 1735  
France 00800 0225 5734  
Germany 0800 1005759  
Italy 00800 0225 5734  
Spain 00800 0225 5734  
UK 00800 0225 5734  
USA & Canada 1 800 228 5166

**EU REP**

SDI Germany GmbH  
Hansestrasse 85  
51149 Cologne, Germany  
T: +49 2203 9255 0

Last Revised: 08-2025

## ENG INSTRUCTIONS FOR USE

Riva Coat is designed for the prevention of saliva and water contamination to exposed surfaces of glass ionomer cements during the first 24 hours immediately after placement.

### INDICATIONS

- Glass ionomer cement restorations requiring a coating material after placement.

### CONTRAINDICATIONS

- Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

### INSTRUCTIONS FOR USE

1. Place the glass ionomer cement according to manufacturer's instructions.
2. When restoration has hardened, apply Riva Coat to all exposed surfaces of restoration.
3. Light cure for 20 seconds.
4. Replace cap immediately after use.

### STORAGE AND HANDLING

- Material is light sensitive. Do not dispense Riva Coat until ready to use.
- Store in a dark place at temperatures between 2° and 25°C (35° - 77°F). Use at room temperature between 20° and 25°C (68° - 77°F).

### PRECAUTIONS

- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at [www.sdi.com.au](http://www.sdi.com.au) or contact your regional representative.

### FIRST AID

- **Eye (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol / acetone. Wash thoroughly with water.
- **Ingestion:** Drink plenty of water. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.

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