seT PP

ENG

SELF ETCHING, SELF ADHESIVE RESIN CEMENT INSTRUCTIONS FOR USE

'seT PP' is a fluoride releasing self etching, self adhesive dual-cure resin cement in a paste/paste system. 'seT PP' is designed for the cementation of indirect ceramic, metal or composite inlays, onlays, crowns, bridges, posts and screws.

COMPOSITION:
35% by weight methacrylate ester.
65% by weight inorganic filler.

INDICATIONS FOR USE:
Ceramic, metal or composite inlays, onlays, crowns, bridges, posts, and screws.

CONTRA-INDICATIONS:
Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

INSTRUCTIONS FOR USE:

RESTORATION PREPARATION:
Clean thoroughly and prepare the restoration according to manufacturer's instructions.

TOOTH PREPARATION:

1. Select the appropriate shade of 'seT PP'.
2. Remove the temporary restoration and clean the preparation thoroughly using an oil free pumice to remove any remaining temporary cement (NOTE: temporary restoration should only be seated with eugenol-free cement).
3. Try in the restoration.
4. Again, clean with water and dry, but DO NOT DESICCATE. WARNING: In cases where pulp protection is necessary, use a hard setting calcium hydroxide liner before the final impression is made. Substances such as hydrogen or carbamide peroxide, desensitizers, disinfectants, astringents and dentin sealants etc should not be used as residues may interfere with bond strength and setting reactions.

PREPARATION OF SYRINGE:

Remove cap. Dispense 2-3mm of material onto a pad and discard (to ensure correct ratio of extrusion). Attach a mixing tip to the syringe.

MIXING BY HAND:

Without mixing tip attached, extrude even amount of material onto mixing pad. With a suitable spatula mix the two pastes together for 30 seconds.

APPLICATION OF 'seT PP'

1. Protect the work area against contamination from intra-oral fluids.
2. Place or extrude 'seT PP' evenly onto all bonding surfaces of the restoration and tooth.

NOTE: Do not use Lentulo-Spirals to insert the cement into root canals, as this can excessively accelerate setting. Insert cement into root canal with the appropriate automix endotip and avoid vibrating posts since it can entrap air.

Working time at room temperature	2 minutes
Net setting time at oral temperature	5 minutes
Light-curing time	20 seconds

Note: These times are approximate only, and may vary based on storage conditions, humidity, age of product and temperature. The 20s photopolymerization may not be enough when a restoration is between the curing unit and seT PP.

3. Seat restoration, and remove excesses
4. Leave it untouched for 3 minutes before any light curing is carried out.
5. Light cure thoroughly (minimum 40 seconds per surface). In the case of composite and all ceramic materials, light cure 'seT PP' through the restoration.
6. Polish marginal area.

STORAGE AND HANDLING:

Avoid excess heat. For optimum performance, refrigeration of the product is recommended.

Store at temperatures between 2° and 25°C (35° - 77°F). For optimum freshness, refrigeration is optional.
Use at room temperature between 20° and 25°C (68° - 77°F).

PRECAUTIONS:

For professional use only. Keep out of reach of children. Do not take internally.

Do not use after expiry date.

Caution: Federal Law restricts this device to sale by or on the order of a dentist.

MSDS available at www.sdi.com.au or contact your regional representative..

FIRST AID:

Eye (contact): Wash thoroughly with water and seek medical attention.
Skin (contact): Remove using a cloth or sponge soaked in alcohol.
Wash thoroughly with water.
Ingestion: Drink plenty of water/milk. Seek medical attention if symptoms persist.
Inhalation: No symptoms expected.

PACKAGING:

- 7g Syringes
- Available shades: A1, A2, OA3, Translucent, White Opaque.
- Assorted Kit, containing one syringe each of shades A1, A2, OA3, Translucent, White Opaque shades, 70 mixing tips.
- Mixing tips - 50 pack.

Last Revised: 08-2021

Made in Australia by SDI Limited
3-15 Brunson Street
Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au

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