Made in Australia by SDI Limited
3-15 Brunson Street, Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
UK 00800 0225 5734
USA & Canada 1 800 228 5166

EC

REP

SDI Germany GmbH
Hansestrasse 85
51149 Cologne, Germany
T: +49 2203 9255 0

Last Revised: 09-2024

PAK41346BER

ENG

INSTRUCTIONS
FOR USE

INTENDED USE:

Zipbond Universal is designed for bonding dental materials. Zipbond Universal can also be used as a sealant or protective varnish.

DESCRIPTION:

Zipbond Universal is a single component, fluoride releasing, light cured universal adhesive system designed to provide consistent and reliable bonding using self etch, selective enamel etch, or total etch techniques.

INDICATIONS:

Direct Applications

- Light-cured composite restorations
- Light-cured compomers
- Cavity sealing prior to placement of indirect restorations
- Composite repair of porcelain and hybrid ceramic restorations
- Sealing exposed root surfaces which are causing hypersensitivity
- Core build ups using light-cure/ dual-cure composites
- Protective varnish for glass ionomer (GI) filling

Indirect Applications

- Cementation of indirect restorations

CONTRAINDICATIONS:

1. Pulp capping.
2. Do not use in anyone who has an allergy to acrylics or who has an allergic reaction to the material of this product.
3. Not to be used in combination with eugenol materials or haemostatics containing ferric compounds. Do not use substances such as desensitisers, disinfectants, astringents, dentine sealants, rinsing solutions containing EDTA or hydrogen peroxide. The residues of these substances can impair the bond strength and setting reaction of the adhesive, particularly with self-cure and dual cure materials.
- 4.

SIDE EFFECTS:

- Areas around the pulp should be protected with a suitable pulp/ dentine protector, such as a calcium hydroxide liner, to prevent any potential irritation.
- The components of Zipbond Universal could, on rare occasions, cause sensitisation. In these cases, Zipbond Universal should not be used.

INTERACTIONS:

- Eugenol containing substances may inhibit the polymerisation process of adhesive materials. Therefore, using such materials in conjunction with Zipbond Universal must be avoided.

INSTRUCTIONS FOR USE:

Preparation:

Isolation techniques must be used to prevent contamination. Rubber dam is the preferred mode of isolation. In cases where pulp protection is necessary, use a calcium hydroxide liner.

Tooth Preparation

1. Remove loose preparation debris by spraying with water.
2. Gently air-dry using 2-3 bursts of water-free/oil-free air or cotton pellets. Do not over dry the cavity. The cavity surface should have a slightly glossy appearance (do not desiccate). Over drying can lead to post-operative sensitivity.

DIRECT ADHESIVE APPLICATION

Direct filling restorations using light-cured composite and compomers/ Cavity sealing prior to placement of indirect restorations.

Self-Etch Technique

1. Shake bottle prior to removing cap. Dispense one or two drops (depending on restoration size) of Zipbond Universal into a mixing well. Immediately replace the cap.
- CAUTION:** Do not dispense the material directly into the patient's mouth.
- NOTE:** Proceed to the next steps quickly as the liquid can evaporate and result in poor bonding.
2. Apply adhesive into the cavity by scrubbing for 10 seconds with a brush, such as Points (SDI Limited).
3. Leave on cavity surface for 10 seconds.
4. Blow with water-free/oil-free air until you see no movement of the bond (minimum 5 seconds).
- NOTE:** A vacuum aspirator can be used to prevent adhesive liquid from scattering.
5. Light cure for 10 seconds using high powered LED curing light (440-480nm wavelength), such as the Radii curing LED range (SDI Limited). Operate the light according to the manufacturer's instructions.
6. Place composite, such as Luna 2 (SDI Limited) or Aura Bulk Fill (SDI Limited), or place indirect restoration, all according to the manufacturer's instructions.
7. Instruct patient not to eat or drink for at least one hour after the procedure.

Selective Etch Technique

1. Apply phosphoric acid etching gel, such as Super Etch 37% (SDI Limited), to tooth enamel according to the manufacturer's instructions.
2. Rinse thoroughly with water and dry with water-free/oil-free air or with cotton pellets. Do not over dry.
3. Apply adhesive according to steps in 'Self-Etch Technique'.

Total Etch Technique

1. Apply phosphoric acid etching gel, such as Super Etch 37% (SDI Limited), to tooth structure (enamel & dentine) according to the manufacturer's instructions.
2. Rinse thoroughly with water and dry with water-free/oil-free air or with cotton pellets. Do not over dry.
3. Apply adhesive according to 'Self Etch Technique'.

Composite repair of porcelain and hybrid ceramic restorations

1. Prepare substrate according to Manufacturer's Instructions For Use.
2. Apply adhesive to the substrate by scrubbing for 10 seconds with a brush, such as Points (SDI Limited).
3. Leave on substrate surface for 10 seconds.

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SDI Holdings Pty Ltd Do Brasil CNPJ 42.943.930/0001-09