# SDI RIVA CONDITIONER

<b>ENG</b>	POLYACRYLIC ACID CONDITIONER
<b>CES</b>	KYSELINA POLYAKRYLOVÁ
<b>DAN</b>	POLYAKRYLSYRE CONDITIONER
<b>DEU</b>	POLYACRYLSÄURE-CONDITIONER
<b>FIN</b>	POLYAKRYYLIHAPPO HOITOAINE
<b>FRA</b>	CONDITIONNEUR D'ACIDE POLYACRYLIQUE
<b>ITA</b>	BALSAMO ACIDO POLIACRILICO
<b>LIT</b>	POLIAKRILLO RUGŠTIES KONDICIONIERIUS
<b>NLD</b>	POLYACRYLZUUR CONDITIONER
<b>NOR</b>	POLYAKRYLSYREBALSER
<b>POL</b>	ODŻYWKA Z KWASEM POLIAKRYLOWYM
<b>POR</b>	CONDICIONADOR DE ÁCIDO POLIACRÍLICO
<b>SPA</b>	ACONDICIONADOR DE ÁCIDO POLIACRÍLICO

	Made in Australia by SDI Limited 3-15 Brunsondon Street, Bayswater, Victoria 3153, Australia
Australia 1 800 337 003 Austria 00800 0225 5734 Brazil 0800 770 1735 France 00800 0225 5734 Germany 0800 1005759 Italy 00800 0225 5734 Spain 00800 0225 5734 UK 00800 0225 5734 USA & Canada 1 800 228 5166	 <p>SDI Germany GmbH Hansestrasse 85 51149 Cologne, Germany T: +49 2203 9255 0</p>
PAK41132 J	     <p>Last Revised: 10-2025</p>

## ENG INSTRUCTIONS FOR USE

Riva Conditioner is a mild solution of polyacrylic acid developed to partially remove the smear layer of dentin and enamel. This optimizes adhesion of the glass ionomer cement to tooth structure.

#### INDICATIONS:

Glass ionomer cement restorations requiring a conditioning material prior to application.

#### CONTRAINDICATIONS:

Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

#### INSTRUCTIONS FOR USE:

- After tooth preparation, apply Riva Conditioner to the prepared surfaces.
- Leave for 10 seconds.
- Rinse thoroughly with water.
- Remove excess water. Keep moist. Avoid contamination.

**IMPORTANT:** Be careful not to completely dry the surface.
- Apply glass ionomer according to manufacturer’s instructions.
- Replace cap immediately after use.

#### STORAGE AND HANDLING

- Do not dispense Riva Conditioner until ready to use.
- Store in a dark place at temperatures between 10° and 25°C (50° - 77°F).
- Use at room temperature between 20° and 25°C (68° - 77°F).

#### PRECAUTIONS:

- For professional use only.
- Keep out of reach of children.
- Do not take internally.
- Do not use after expiry date.
- Caution:** Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

#### FIRST AID

- Eye (contact):** Wash thoroughly with water and seek medical attention.
- Skin (contact):** Wash thoroughly with water.
- Ingestion:** Drink plenty of water/milk. Seek medical attention if symptoms persist.
- Inhalation:** No symptoms expected.

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