SDI poladay CP
35% carbamide peroxide
advanced tooth whitening system
INSTRUCTIONS FOR USE
FOR PATIENTS AND
DENTAL PROFESSIONALS

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ESPAÑOL

Pola Day CP is a 35% carbamide peroxide, high viscosity, neutral pH take home tooth whitening gel. It is a pleasantly flavoured, crystal clear sustained release gel with a water content high enough to reduce sensitivity.

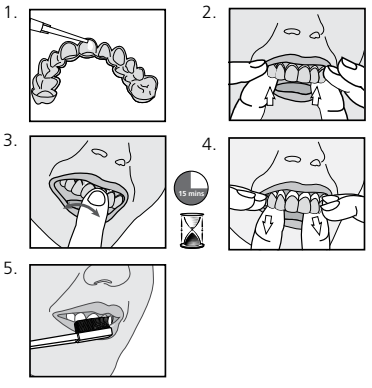
- INDICATIONS FOR USE:**
- Whitening of discoloured vital teeth.
 - Whitening of discoloured non-vital teeth.

COMPOSITION: 35% Carbamide peroxide
40% Water
0.5% Flavour
24.5% Thickening agents

NET WEIGHT:
Depending on kit purchased, 1.3g syringes or 3g syringes. Please refer to outside of box.

HYDROGEN PEROXIDE EQUIVALENT:
Pola Day CP 35% Carbamide Peroxide = 11.7% Hydrogen Peroxide

- INSTRUCTIONS:**
- Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
 - Seat the tray, with the gel around the teeth.
 - Wipe away excess gel in mouth with a tissue or dry soft brush.
 - After 15 minutes of treatment, remove tray. Rinse tray and mouth with lukewarm water.
Note: Treatment time can be up to 30 minutes if required.
 - Brush teeth.

IN-OFFICE JUMP START (FOR DENTAL PROFESSIONALS)
When used as a jump start in the dental office, treatment time is 30 minutes.

- STORAGE AND HANDLING:**
- Store product in the refrigerator (2°-8°C / 35°-45°F).
 - Do not use after expiration date.
 - Use product at room temperature.

- PRECAUTIONS:**
- To be used under the supervision of a dentist.
 - Do not eat, drink or smoke during treatment.
 - Do not smoke immediately after treatment - wait for at least two hours.
 - Food and drinks containing strong colours should be avoided or consumed in moderation during the entire treatment period.
 - Keep out of reach of children and pets.
 - Not to be used by pregnant or lactating women or children under 14 years old.
 - Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using this product.
 - Discontinue use if any unusual sensitivity or reactions occur and consult your dentist.
 - Do not inject Pola Day CP into the body.
 - CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST.**
 - MSDS available at www.sdi.com.au or contact your regional representative.

- FIRST AID:**
- Eye (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
 - Skin/Tissue (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
 - Ingestion (large amount):** Drink plenty of milk or water and seek medical advice.

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