ENG ADVANCED TOOTH WHITENING SYSTEM
FRA HAUTE TECHNOLOGIE DANS LE BLANCHIMENT DES DENTS
POR SISTEMA AVANÇADO DE CLAREAMENTO DENTAL
SPA SISTEMA AVANZADO DE BLANQUEAMIENTO DENTAL

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Made in Australia by SDI Limited
 3-15 Brunston Street, Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
 Austria 00800 0225 5734
 Brazil 0800 770 1735
 France 00800 0225 5734
 Germany 0800 1005759
 Italy 00800 0225 5734
 Spain 00800 0225 5734
 UK 00800 0225 5734
 USA & Canada 1 800 228 5166
www.sdi.com.au
www.polawhite.com.au

Last Revised: 08-2025

SCAN FOR INSTRUCTION VIDEO

2°C / 35°F
 8°C / 46°F

Allergies & Precautions

ENG INSTRUCTIONS FOR USE

FOR PATIENTS AND DENTAL PROFESSIONALS

Pola Day CP is a 35% carbamide peroxide, high viscosity, neutral pH take home tooth whitening gel. It is a pleasantly flavoured, crystal clear sustained release gel with a water content high enough to ensure maximised patient comfort.

INDICATIONS FOR USE:

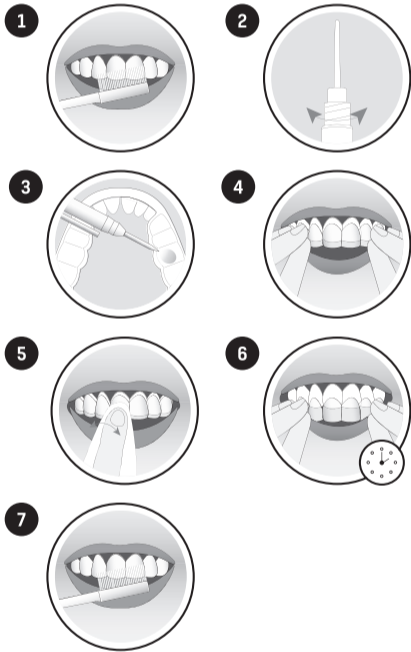
- Whitening of discoloured vital teeth.
- Whitening of discoloured non-vital teeth.

NET WEIGHT:

Depending on kit purchased, 1.3g syringes or 3g syringes. Please refer to outside of box.

INSTRUCTIONS:

1. Brush teeth.
2. Remove the whitening gel syringe cap and attach application tip by twisting it securely onto the syringe.
3. Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
Tip: Use the dosage indicators on the syringe as a guide.
3g syringe - Use approximately one sixth of the syringe per application; this amount is sufficient for both the top and bottom whitening trays.
1.3g syringe - Use approximately one third of the syringe per application; this amount is sufficient for both the top and bottom whitening trays.
4. Seat the tray, with the gel around the teeth.
5. Wipe away excess gel in mouth with a tissue or dry soft brush.
6. After 15 minutes of treatment, remove tray. Rinse tray and mouth with lukewarm water.
 Note: Treatment time can be up to 30 minutes if required.
7. Brush teeth.

IN-OFFICE JUMP START (FOR DENTAL PROFESSIONALS)

When used as a jump start in the dental office, treatment time is 30 minutes.

STORAGE AND HANDLING:

- Store product in the refrigerator (2° - 8°C / 35° - 46°F).
- Do not use after expiration date.
- Use product at room temperature.

PRECAUTIONS:

- To be used under the supervision of a dentist.
- Do not eat, drink or smoke during treatment.
- Do not smoke immediately after treatment - wait for at least two hours.
- Food and drinks containing strong colours should be avoided or consumed in moderation during the entire treatment period.
- Keep out of reach of children and pets.
- Not to be used by pregnant or lactating women.
- Certain countries have age restrictions for whitening products where it is not to be used under a certain age: - New Zealand: under 16 years.
- Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using this product.
- Discontinue use if any unusual sensitivity or reactions occur and consult your dentist.
- Do not inject Pola Day CP into the body.
- SDS available at www.sdi.com.au or contact your regional representative.

WARNING FOR CANADA ONLY

- If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult an oral health professional.
- Products containing peroxides are not recommended for use by children under 12 years of age.
- Consult your oral health professional before prolonged use of this product.
- Avoid swallowing the cosmetic or part thereof.
- Avoid contact of the product with the eye.
- Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.

FIRST AID:

- **Eye (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
- **Skin/Tissue (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
- **Ingestion (large amount):** Drink plenty of milk or water and seek medical advice.

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