



24 CLINICAL EVALUATORS

212 TOTAL USES

90% CLINICAL RATING

Key features: Silver Diamine Fluoride (SDF) and Potassium Iodide (KI)
 • Two-step procedure minimizes risk of staining • Multiple indications

Description

Riva Star is a next generation SDF:

- SDF + KI system
- Unlike other silver diamine fluoride systems, Riva Star's two-step procedure minimizes the risk of staining
- Two-year and beyond desensitizing effect
- Non-invasive and ideal for minimally invasive dentistry

Indications

- Desensitizing
- Caries arrest
- Caries detection
- SMARTer glass ionomer sandwich technique

Unique Attributes

- Versatile indications
- An SDF material with minimal staining
- Increases bond strengths of glass ionomer restorative materials to dentin



Clinician: Dr. Ona Erdt



Clinical Tips

- Very good at determining caries missed at initial excavation.
- It can be tempting to only place one application of the KI material, but make sure to place multiple applications until the white precipitate turns clear (around 90 seconds).

"LESS WORRY ABOUT STAINING."

Evaluators' Comments

"Ease of use and minimal staining. This is a well thought out product."

"It did not turn the unattractive black color like its competitor."

"Much, much better than competitor. Less messy."


"Easy to use two-step procedure."

"Desensitizing effect was great."


"Two steps require longer working time."

"I found it hard to adequately isolate due to the viscosity of this material."

Evaluation Summary:

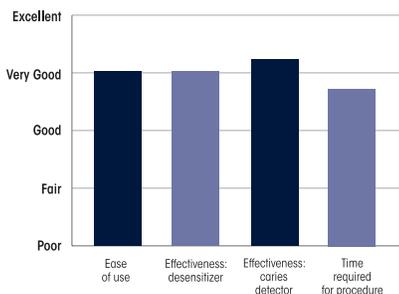

Compared to Competitive Products:

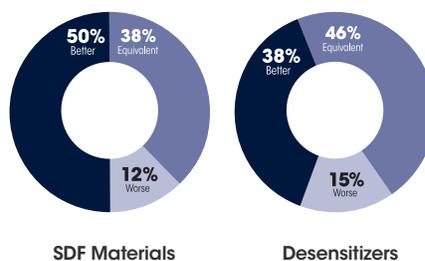

Consultants who would:

- 88%** Recommend to a colleague
- 38%** Yes, instead of current product
- 33%** Yes, in addition to current product
- 21%** I might want to order it for certain cases