SDI

everfix

RESTORATIVE MATERIAL
AMALGAM ALTERNATIVE
INSTRUCTIONS FOR USE

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ESPAÑOL

Everfix is designed for use as an alternative to amalgam restorative materials and is easy to place with tooth like aesthetics. **Everfix** can be used for a wide range of direct restorative applications. Shade: universal posterior shade (A3).

INDICATIONS FOR USE:

- Permanent Class I restorations
- Permanent Class II restorations*
- Primary teeth restorations
- Core build up

*The isthmus needs to be 1-1.5 mm distance from cusp peaks.

CONTRA-INDICATIONS:

- Pulp capping
- Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

PROCEDURE:**PREPARATION:**

1. Isolate, prepare and clean tooth.
Note: A conservative cavity should be prepared, employing standard techniques and instruments. Excessive mechanical retention is not necessary. In cases where pulp protection is necessary, use a calcium hydroxide liner.
2. If a matrix system is used, apply coca butter or petroleum jelly to the inside surface.
3. Rinse thoroughly with water.
4. Remove excess water. Keep surface moist (glistening).
Avoid contamination.

IMPORTANT: Be careful not to completely dry the surface.

MIXING AND PLACING:

Powder / Liquid Ratio (g/g)	0.50 / 0.13
Mixing time	10"
Working time (min., sec.)	1'30"
Setting time (min., sec.)	3'30"
Minimum delivery / capsule (mL)	0.12
Test conditions: Temperature (23 ± 1°C), relative humidity (50 ± 10%) ISO9917-1 (Dental water based cement).	

1. Push the plunger on a hard surface until it is flush with the body of the capsule.
2. **Important:** Do not click with an applicator before you mix.
Immediately place activated capsule into the Ultramat 2 triturator or any high speed capsule mixer (4000 – 4800 rpm).
3. Mix for 10 seconds.
4. Immediately remove the **Everfix** capsule and place into the capsule applicator.
5. Click the trigger of the capsule applicator until **Everfix** paste is seen through the clear nozzle.
6. Extrude **Everfix** into the cavity, pack and contour. Being careful not to trap air under the restoration.
NOTE: Temperatures higher than 23±1°C will shorten the working time specified in the above table.
7. At the end of 3:30 min setting time, finish restoration under water spray using superfine diamond burs.
8. Use a probe to separate and remove any matrix used.

STORAGE AND HANDLING:

- Do not remove **Everfix** capsules from their foil packets until ready to use.
- Store at temperatures between 4°C - 25°C (39°F - 77°F).
- Use at room temperature between 20°C and 25°C (68°F - 77°F).

PRECAUTIONS:

- For professional use only.
- Keep out of reach of children.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- SDS available at www.sdi.com.au or contact your regional representative.

FIRST AID:

- **Wash (contact):** Wash thoroughly with water and seek medical attention.
- **Skin (contact):** Remove using a cloth or sponge soaked in alcohol. Wash thoroughly with water.
- **Ingestion:** Rinse mouth thoroughly. Drink plenty of water/milk. Seek medical attention if symptoms persist.
- **Inhalation:** No symptoms expected.

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Made in Australia by SDI Limited
SDI Germany GmbH