



**SDI**


# RIVA SELF CURE/HV

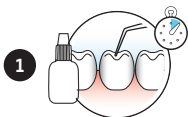


Isolate tooth, prepare cavity.  
**OPTIONAL:** Apply Riva Conditioner for 10 seconds, wash and remove excess water.

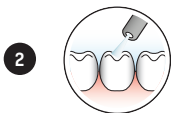

Wash thoroughly.

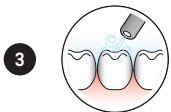


Remove excess water.  
Keep moist.

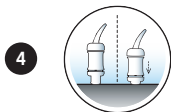


Activate the capsule by pressing the plunger on to a hard surface until it completely locks inside the body of the capsule (ring on the plunger is flush with the body).  
Only activate prior to mixing.




Immediately mix in a triturator for 10 seconds.

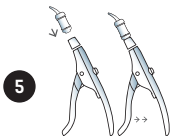


Immediately place into capsule applicator and click trigger until paste is seen through the nozzle.

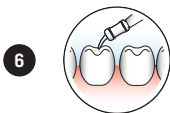

Extrude Riva Self Cure into cavity in a single increment.

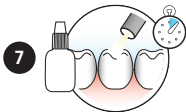

Apply Riva Coat and light cure for 10 seconds.

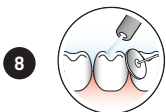



Commence final finishing under water spray.



Apply Riva Coat and light cure for 10 seconds.





1

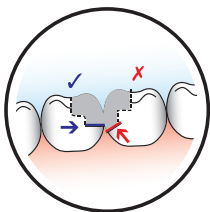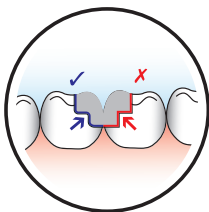Rounded internal angles and  
horizontal gingival wall.

2

No over contours.

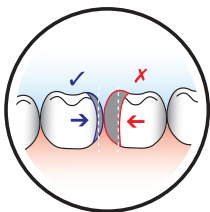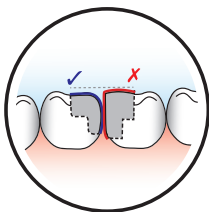

3

Rounded external contour, from  
1 to 1.5mm away from cusp peaks.